EN Emiluma™ Transparent Silicone for IDB

Description/Indications:
Emiluma™ is a transparent silicone to be used as a delivery tray for indirect orthodontic bonding and can be used with any bondable bracket, tube or lingual attachment.

Procedure:
1. Load the cartridge in the dispensing gun.
2. Immediately prior to applying the silicone, remove the cartridge cap and set aside as it can be replaced.

3. Gently squeeze the dispenser to express enough material from the cartridge to ensure even flow of the base and catalyst. (About ¼ inch of material).
Note: Always express a small amount from the cartridge before each use.

4. Wipe clean and attach the mixing tip to the cartridge.
5. Squeeze the dispenser to express Emiluma onto the model.

6. Dispense a small, consistent amount of Emiluma directly on the bracket. The amount should be enough material to ensure that each orthodontic bracket is completely enclosed.

7. If employing the Quick Indirect Bonding System (IDBS™) technique, place a thick layer of a firm, clear silicone (like Memosil or Odontosil) directly over the Emiluma silicone before Emiluma has set up. This second layer of silicone provides structural support to the tray and facilitates precise placement of the orthodontic brackets.

8. Follow remaining steps in the Quick IDBS™ technique online: www.opalorthodontics.com

9. If employing the vacuum-formed indirect bonding tray technique, use Emiluma silicone and then form a vacuum-formed hard plastic outer shell for structural support. The steps in this technique would replace #7 and #8 above.

10. After use of Emiluma, store cartridges standing upright with the used mixing tips attached to protect the remaining materials from contamination.
PLEASE NOTE:

• The Quick Indirect Bonding System or Quick IDBS™ was developed by Dr. Masatada Koga and Dr. Kazuya Watanabe of Tokyo, Japan.
• The temperature of Emiluma affects the working and setting times. Ideally it should be kept at room temperature (18-25° C) or (65-77° F).
• Avoid contaminating Emiluma with chemicals that could inhibit the set (e.g. acrylic and methacrylate resins, latex rubber and sulphur compounds).

AUXILIARY PRODUCTS IN A QUICK IDBS™ TECHNIQUE:
Emiluma is one product in a two silicone tray technique – commonly referred to as the Quick IDBS™ technique. See www.opalorthodontics.com for other components in the Quick IDB System.

DISPENSING GUN COMPATIBILITY:
Emiluma is compatible with the Emiluma II dispensing gun.
DISPENSING GUN MAINTENANCE:
The Ultradent Products Impression Gun can be sterilized using heat sterilizers, including STATIM Autoclaves, conventional autoclaves, chemical vapor or dry heat sterilizers.

CONTRA-INDICATIONS:
There are no known contra-indications provided when the product is used as directed. There are no known harmful reactions or secondary effects on patients and/or dental personnel using Emiluma transparent silicone.

IMPORTANT NOTES AND PRECAUTIONS:
1. The temperature of Emiluma affects working time. Ideally it should be kept at room temperature between 18-25° C or (65-77° F). Even with those boundaries, changes in temperature will affect the working time. A 2° C temperature increase will shorten working time 10 – 15 seconds; conversely, a 2° C temperature decrease will lengthen working time 10 – 15 seconds. You may refrigerate to increase working and setting times. All testing for working and setting times was done at 22° C / 72° F.
2. Shelf life is 36 months.

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© Copyright 2018 Ultradent Products, Inc. All Rights Reserved.