Universal Dentin Sealant Protective Barrier
---

Description:

Universal Dentin Sealant is a biocompatible, non-polymerizable, high molecular weight resin in an ethanol carrier.

Indications for Use

Universal Dentin Sealant seals exposed dentin tubules against harsh irritating resins, acids, cements, and restorative chemicals and can be used as a conventional cavity varnish under amalgam restorations. It is recommended for alleviating post-op root planning and scaling sensitivity.

Directions for Use:

- Remove Luer Lock cap.
- Securely attach brush tip.
- Thoroughly isolate and dry area.
- Verify flow of Universal Dentin Sealant prior to applying intraorally.
- For sensitive root surfaces after scaling or for protecting root surfaces from harsh chemicals, apply a thin coat of Universal Dentin Sealant to area and gently air dry for 5 - 10 sec.
- When using as a cavity varnish under amalgam restorations, paint the Universal Dentin Sealant on the cavity floor and walls avoiding the cavosurface margin, then air dry for 5-10 seconds.

Precautions and Warnings:

- Carefully read and understand all instructions before using Universal Dentin Sealant.
- For professional use only.
- Use only as directed.
- Store at room temperature.
- Resins can be sensitizing. Confirm patient has no known allergies to treatment materials. Avoid repeated contact of uncured dental resin with skin. Do not use on patients with a known sensitivity to acrylates or other resins. If allergic reaction, dermatitis or rash develops, consult a physician.
- Always verify material flow prior to applying intraorally. If resistance is met, replace tip and re-check.
- Isolate strong chemicals to area of treatment.
- Chemical may set in tip after a few hours. To avoid clogging, remove tip and re-cap with Luer Lock Caps between each use.
- Do not use after expiration date noted on containers.
- Re-cap, disinfect and wipe syringe with an intermediate level disinfectant between uses. If disposable syringe cover is used, remove tip, re-cap, and discard syringe cover.
- Dispose of used tips and empty syringes properly.
- Tips are single use only and disposable. To avoid cross contamination, do not re-use tips.
- Keep products out of direct heat/sunlight.
- Keep out of reach of children.
- For SDS and additional information about using Universal Dentin Sealant and related products, please go to www.ultradent.com

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