

Opalescence



Tooth Whitening

ULTRADENT

Powerful, professional whitening to go!

Opalescence Go whitening features ready-to-go convenience with the UltraFit™ tray for an even more adaptable and comfortable whitening experience.

- Unique tray material offers a remarkably comfortable fit and easily conforms to any patient's smile
- Molar-to-molar coverage ensures the gel comes in contact with more posterior teeth
- UltraFit tray's superior adaptation ensures the maximum amount of gel stays in contact with the teeth during whitening
- Available in 10% hydrogen peroxide (with wear times from 30–60 minutes) and 15% hydrogen peroxide (with wear times from 15–20 minutes)





Additional features of Opalescence Go whitening

- Mint and Melon flavors
- Convenient prefilled trays can be worn right out of the package
- Optimal gel quantity allows easy cleanup after whitening
- Every patient kit includes a 1 oz. tube of Opalescence™ Whitening toothpaste

Opalescence Go Patient Kits Case of 6

Flavor	10%	15%
Mint	4635-US	4638-US
Melon	4636-US	—

10 x Each upper/lower trays
and a 1-ounce tube of toothpaste in each kit



Opalescence Go Mini Kits Case of 12

Flavor	10%	15%
Mint	4645-US	4648-US
Melon	4646-US	—

4 x Each upper/lower trays in each kit



Opalescence Go Sample Dispenser Kits

Flavor	10%	15%
Mint	4893-US	4894-US
Melon	4890-US	—

20 x Each upper/lower trays in each kit
20 x Patient instructions



Instructions



1. Remove product from packaging.
"U" - Upper whitening tray
"L" - Lower whitening tray



2. Position upper tray on teeth.



3. Bite firmly, then suck on tray for 2 seconds.



4. Remove colored outer tray, leaving the white inner tray on teeth. Repeat the process for the lower tray.



5. After indicated wear time, remove whitening trays and brush teeth.

Refrigeration is recommended for all Opalescence Go whitening products.

