

Opalescence



Tooth Whitening

ULTRADENT

Powerful,
professional whitening
to go!





Opalescence Go whitening features ready-to-go convenience with the UltraFit™ tray for an even more adaptable and comfortable whitening experience.

- Unique tray material offers a remarkably comfortable fit and easily conforms to any patient's smile
- Molar-to-molar coverage ensures the gel comes in contact with more posterior teeth
- UltraFit tray's superior adaptation ensures the maximum amount of gel stays in contact with the teeth during whitening
- Available in 10% hydrogen peroxide (with wear times from 30–60 minutes) and 15% hydrogen peroxide (with wear times from 15–20 minutes)
- Every patient kit includes a 1 oz. tube of Opalescence™ Whitening toothpaste

Opalescence Go Patient Kits Case of 6

Flavor	10%	15%
Mint	4635-US	4638-US
Melon	4636-US	—

10 x Each upper/lower trays
and a 1-ounce tube of toothpaste in each kit



Opalescence Go Mini Kits Case of 12

Flavor	10%	15%
Mint	4645-US	4648-US
Melon	4646-US	—

4 x Each upper/lower trays in each kit



Opalescence Go Sample Dispenser Kits

Flavor	10%	15%
Mint	4893-US	4894-US
Melon	4890-US	—

20 x Each upper/lower trays in each kit
20 x Patient instructions



Refrigeration is recommended for all Opalescence Go whitening products.

Instructions



1. Remove product from packaging.
"U" - Upper whitening tray
"L" - Lower whitening tray



2. Position upper tray on teeth.



3. Bite firmly, then suck on tray for 2 seconds.



4. Remove colored outer tray, leaving the white inner tray on teeth. Repeat the process for the lower tray.



5. After indicated wear time, remove whitening trays and brush teeth.

