



4757 - Transcend **UB Singles 1pk**
10 x 0.2 g singles Universal Body shade

4817 - Transcend **UB Singles 4pk**
40 x 0.2 g singles Universal Body shade



4733 - Transcend **UB Syringe 1pk**
1 x 4 g syringe Universal Body shade

4734 - Transcend **UB Syringe 4pk**
4 x 4 g syringes Universal Body shade

NEW



5876 - Transcend Bulk Fill Flowable
UB Syringe Refill
1 x 2 g syringe Universal Body shade
10 x Micro 20 ga tips

Complementary Shades



Transcend **Syringe 1pk**
4 g syringe

Dentin	1pk	Enamel	1pk
A1D	4727	EN	4731
A2D	4728	EW	4732
A3D	4729		
B1D	4730		



Transcend **Singles 1pk**
10 x 0.2 g singles

Dentin	1pk	Enamel	1pk
A1D	4744	EN	4748
A2D	4745	EW	4749
A3D	4746		
B1D	4747		



4726 - Transcend **Syringe Intro Kit**
1 x 4 g syringe of each shade:
A1D, A2D, A3D, B1D, EN, EW, UB



4814 - Transcend **Singles Intro Kit**
10 x 0.2 g singles of each shade:
A1D, A2D, A3D, B1D, EN, EW, UB

Pair Composite Wetting Resin with any Ultradent composite to improve instrument and composite glide when sculpting and contouring.



3059 - Composite Wetting Resin **Syringe 2pk**
2 x 1.2 ml syringes



SCAN
TO VIEW
PRICING

TRANSCEND[™]
COMPOSITES

Beautiful Restorations with Just One Shade



Patient was getting married and wanted whiter, more esthetic teeth. Due to the time constraint, the patient opted to do full composite veneers from 7-10 with cusp buildups on both canines prior to whitening. I used Transcend universal composite in Universal Body shade and then whitened after restorations were completed.

Courtesy of Dr. Jaleena Fischer-Jessop, Chief Clinical Officer of Ultradent Products, Inc.



NEW

Simply Beautiful Results for Any Restoration RESIN PARTICLE MATCH[™]


Thanks to Ultradent's Resin Particle Match[™] technology, the refraction indices of the resin and particles work together to allow Transcend composites to blend with the surrounding tooth color. For many cases, you can use just one Universal Body shade and no blocker without sacrificing beautiful results.



One-Shade Restoration


Patient presented with a failing composite restoration. Filling was removed, which revealed tattooing and a mesial crack. The mesial crack was still a small, manageable size and therefore a direct bonded restoration could be done instead of an indirect crown restoration. Products used: Ultra-Etch[™] etchant, Consepsis[™] solution, and Peak[™] Universal Bond adhesive. A very thin layer of A3 PermaFlo[™] flowable composite was used for the initial layer. Transcend composite Universal Body shade was used to restore the tooth. Composite was adjusted according to occlusion and polished up beautifully.

Dr. Jaleena Fischer-Jessop
Chief Clinical Officer of Ultradent Products, Inc.

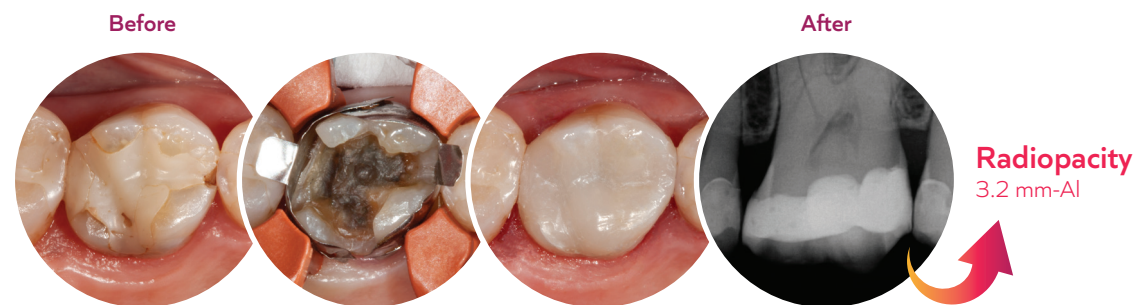

Extra Shades for Demanding Anterior Cases

In addition to the Universal Body shade, Transcend universal composite also includes four dentin shades and two enamel shades for cases when a layering technique is appropriate. These layering shades were handpicked to include only the most popular shades from our esthetic composite lines to ensure you have the shades needed to accomplish restorative work done on a regular basis.

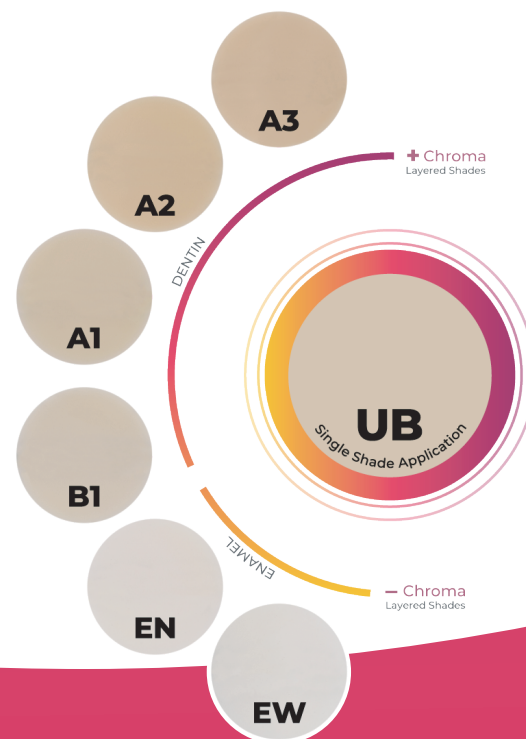

Save Time with Transcend Bulk Fill Flowable Composite

Simplify your restorative process by eliminating tedious layering and repetitive curing. Transcend Bulk Fill flowable composite combines Resin Particle Match[™] technology with a 4 mm depth of cure for efficient procedures. With its deeper depth of cure, Transcend Bulk Fill composite allows you to fill the bulk of a restoration in just one layer, and its excellent adaptability results in less time manipulating the composite. You and your patients will appreciate faster procedures with fewer repetitive steps.

Bulk Fill Restoration

Patient presented with restorations that needed to be replaced. Previous restoration and decay were removed, and the tooth was restored using Transcend Bulk Fill flowable composite with final occlusal layer using Transcend universal composite in Universal Body shade.



Dr. Jaleena Fischer-Jessop
Chief Clinical Officer of Ultradent Products, Inc.

NEW

UP TO
4 mm
DEPTH OF CURE

Perfect Flowability

Transcend Bulk Fill flowable composite seamlessly adapts to the prep as it flows, minimizing instrumentation and simplifying the procedure.



Transcend the Competition

Scan this QR code to see how Transcend composite and its Resin Particle Match technology is the best choice for clinicians who seek to improve their productivity and simplify their inventory control while providing excellent esthetic outcomes to their patients.