Porcelain Etch & Silane EN

Product Description
UltraDent™ Porcelain Etch is a buffered, gelated hydrofluoric acid to be judiciously used for intraoral or extraoral etching of porcelain. UltraDent Silane is a mono-component solution for application to porcelain following etching with Porcelain Etch.

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Forms
Porcelain Etch
• 1.2 ml delivery syringe
• Delivery tip:
◦ Inspiral™ Brush tip
• Silane
• 1.2 ml appliceringsprøtze
• Delivery syringe:
◦ Black Mini™ Brush tip

Indications for Use/Intended Purpose
Porcelain Etch is to be used on etched fractured dental porcelain and for etching porcelain veneers, crowns or inlays in the office. Also use it for etching porcelain crowns before bonding or on orthodontic brackets.
Contraindications
• For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have patient consult their physician.

Warnings and Precautions

Hydrofluoric acid gives off a vapor that irritates respiratory passageways and on a chronic basis can cause damage. UltraDent Porcelain Etch is buffered, gelled, and dispensed in a way that minimizes these problems. Instructions must be followed to ensure complete removal of any harmful material. Silane is applied after etching in preparation for bonding resin. If proper precautions are followed, porcelain may be safely etched intraorally for bonding.

If etching vapors cause irritation, patient must wear protective eyewear with side shields when handling or applying these products
• Wear protective gloves when handling these products.
• Avoid contact with soft tissue, dentin, and enamel.
• If etchant causes any soft tissue, IMMEDIATELY rinse area with copious amounts of water.

• Tips are single use only.
• Use only recommended tips.
• To avoid clogging, remove tip and re-cap with Luer Lock Caps between each use. If tip is ingested, drink large amounts of water and contact a physician. If the product is aspirated, seek immediate medical attention.

Stepwise Instructions
Preparation
1) Freshen the porcelain surface to be repaired with a high-speed diamond bur. If a previous repair is present, remove all old composite to a fresh porcelain surface.
2) Remove all old restorative tissues and restorations using a rubber dam or gingival protective resin barrier.

3) Attach recommended tip and verify flow, color, and consistency of the material inside the mouth. If there is resistance to flow, attach a new tip and test again before using on patient

Use
1) Express an even coating of Porcelain Etch onto prepared porcelain using the recommended tip. Etch porcelain for 90 seconds.

NOTE: If hydrofluoric acid comes into contact with dentin or enamel surfaces, it will increase the porosity and cause discoloration of these surfaces. Therefore, it is important to etch any adjacent tooth structure that may be next to or included in the repair with phosphoric acid before etching the porcelain surfaces.

4) Use the high volume suction for the full 90 second etch time for removal of any etching vapors while acid reacts on the porcelain.

5) Suction again from the surface and thoroughly rinse and dry the surface.

6) Porcelain should have a frosty appearance. If it does not, repeat porcelain etch step.

7) Apply Ultra-Etch™ for 5 seconds to remove porcelain salts and debris formed from the etching process.

8) Gently brush silane into the porcelain with a recommended tip and let sit for 60 seconds. If not completely dry after 60 seconds, air dry with oil-free air.

9) Use bonding agent and restorative materials of choice.

Processing

Storage and Disposal
Store according to outer package label.
Dispose of waste according to local rules, guidelines, and regulations.

Miscellaneous Information

Report any serious incident to the manufacturer and the competent authority.
www.ultradent.com / 1.800.552.5512 / 801.572.4200

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