TriAway™


**Syringe Adapter/DE-Spritzen-Adapter/FR-Adaptateur pour Seringue/
NL-Adapter voor driefunctiespuit/IT-Adattatore per siringhe/ES-Adaptador
de Jeringa/PT-Adaptador de Seringa/SV-Sprutadapter/DA-Adapter til
sprøjte/FI-Ruiskuadapteri/EL-Προσαρμογέας αεροσύριγγας**

Black Micro Tip

White Mac Tip

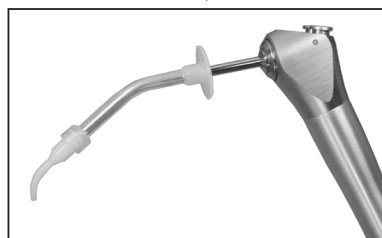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DUTCH

GREEK

U.S. federal law restricts this device to sale by or on the order of a dentist.

Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line,
refer to Ultradent's catalog or call Toll Free 1-800-552-5512.

Outside U.S. call (801) 572-4200.

TriAway™ Syringe Adapter

Manufactured by:
Ultradent Products Inc.
505 West Ultradent Drive (10200 South)
South Jordan, Utah 84095 USA
Made in the USA from globally sourced materials

ULTRADENT
PRODUCTS, INC.

© Copyright 2025 Ultradent Products, Inc. All Rights Reserved. 23879AR11 051625

TriAway™ Syringe Adapter

Description:

The TriAway™ syringe is a disposable irrigator/dryer to be used with Ultradent™ delivery tips on three-way syringes. It will fit most three-way syringe tips with an outside diameter of 0.1523"/3.86mm.

Indications:

Use the TriAway syringe for controlled application of water and/or air to depths of tiny cavity preparations, whether they are minimally invasive operative-type preparations or endodontic preparations. Removal of water or excess bonding agents from even acute corners and line angles of larger preparations occurs with precision and superior predictability and quality.

Procedure:

1. Slip TriAway snugly over three-way syringe tip.
2. Attach Ultradent syringe tip of choice firmly to end.
3. Experiment with air/water, air or water flow on inert surface prior to use in mouth.

Precautions:

1. Do not force compressed air or water into endodontic canals or surgical sites.
2. Tips with a smaller diameter than 0.055" ID should only be used with air or water - not both at the same time.

Use of air/water with smaller diameter tips will not allow flow of both and may lead to backflow.

3. To prevent unwanted dislodgement, verify that all components are firmly attached.
4. Use light to moderate pressure with smaller diameter tips. Pneumatization of tissues may occur if tips are directed into sulcular tissues or surgical sites.
5. Insert TriAway to end of three-way syringe tip.
6. Apply petroleum jelly to three-way syringe tip before inserting TriAway.
7. Do not autoclave the TriAway syringe.

Fig. 1 The White Mac™ (shown here) provide a more predictable air water flow although access into micro preparations may be limited.