

## Disposal


Dispose of waste according to local rules, guidelines, and regulations.

## Miscellaneous Information

- Hypocalcified areas (white spot lesions), which may not be visible to the naked eye, will whiten faster (thereby becoming more obvious during whitening). Continue whitening treatment until the unaffected tooth surface blends. Re-evaluate two weeks after whitening treatment when tooth color has stabilized.
- For a small percentage of the population, their teeth will not appreciably whiten.
- Some amalgam fillings may leave a dark purple color in your whitening tray. This is normal.
- Teeth can re-stain over time. So, do touch up teeth whitening treatments as necessary.

Report any serious incident to the manufacturer and the competent authority. [www.ultradent.com](http://www.ultradent.com) / 1.800.552.5512 / 801.572.4200

Consult your healthcare professional for any product related questions.

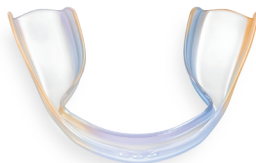

# Opalescence™

Home

Advanced

Dentist-designed at- **Home** whitening

# Opalescence™




## Tooth Whitening Systems

## Welcome to Your Seriously Brighter Smile

Achieve a radiant, confident smile that lights up your world with Opalescence Home Advanced whitening.

Opalescence Home Advanced gives you fast teeth whitening results that are seriously effective. The prefilled UltraFit™ trays are ready to go straight out of the box and conform precisely to your teeth, ensuring the whitening gel stays exactly where it belongs. It makes for a comfortable whitening treatment you can do when you want, where you want.

 Manufactured by Ultradent Products, Inc. 505 West Ultradent Drive, South Jordan, Utah 84095. [www.opalescence.com](http://www.opalescence.com)

Made in USA from U.S. and globally sourced components.

© 2024 Ultradent Products, Inc. All rights reserved.  
1012238AR01 090524



## Instructions



1.

Remove product from packaging.  
'U' – Upper whitening tray  
'L' – Lower whitening tray



2.

Position upper tray on teeth.



3.

Bite firmly, then suck on tray for 2 seconds. Do not press tray against teeth as it pushes the whitening gel against the gums, causing tissue irritation.



4.

Repeat the process for the lower tray.



5.

After indicated wear time, remove whitening trays and brush teeth.

## Wear Times

 10%: 30–60 min

 15%: 15–20 min



Do not re-use to avoid cross contamination



Keep away from heat/sunlight



Store in a cool place



See instructions



Keep out of reach of children

## Product Description

Opalescence™ Home Advanced whitening is a take-home whitening system dispensed in pre-loaded disposable trays. It is a clear, flavored, high viscosity, sticky, hydrogen peroxide gel with potassium nitrate and sodium fluoride.

For all products described, carefully read and understand all instructions and SDS information prior to use.

## Precautions

- Limit contact with soft tissue.
- Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow.
- You may experience temporary discomfort of the gums, lips, throat, tongue, or teeth. Should any of these symptoms persist or worsen after discontinuing use, or in cases of severe discomfort, call your dental professional.
- Avoid swallowing large amounts of gel. Ingestion of small amounts of gel during use is expected.
- If the product is ingested, drink large amounts of water and contact a physician. If the product is aspirated, seek immediate medical attention.
- Restorative materials will not whiten. We recommend whitening dark teeth before restorative placement.
- Wait at least 7–10 days after whitening procedures before any dental restorative work is done.
- Those with serious health concerns should consult their primary care provider prior to treatment.
- Avoid contact with eyes. Rinse immediately if product comes into contact with them.
- To prevent additional irritation, gingival and general oral health should be evaluated prior to initiating treatment.
- Monitor whitening progress to prevent whitening beyond the desired shade or translucency.
- Do not eat, drink, or use tobacco products while whitening trays are in place.
- Do not wear whitening tray while sleeping.
- Bring treatment tray to room temperature before use.
- Pregnant or nursing women should not use teeth whitening products.
- For users with allergy concerns, refer to product allergen document available at [www.ultradent.com](http://www.ultradent.com). If an allergic reaction is observed, rinse exposed area thoroughly with water and consult your physician.