

## Cleaning and Sterilization Processing Instructions

To obtain a copy of these instructions call +1 801 553 4507

Initial treatment at the time of use	Sterilization is required before every use	
Preparation before cleaning	Wipe device with 70% IPA soaked gauze	
Cleaning	Place device in an ultrasonic cleaner with the detergent listed below:	
	<b>Detergent</b>	<b>Cleaning Time</b>
	Henry Schein General Purpose Cleaner or equivalent product	2-10 mins

Rinsing	Rinse device with warm water to remove detergent for 1-2 mins
Drying	1. Dry with gauze. 2. Air dry 30 mins
Maintenance, inspection and testing	Visually inspect that the device is not bent or damaged and free of debris. If debris is present repeat cleaning process
Packaging	Place selected device in approved sterilization packaging (ISO11407 & ISO11607)

Sterilization	Place sterilization package in autoclave using the following settings:			
	Cycle Type	Minimum Sterilization Exposure Time	Minimum Sterilization Exposure Temperature	Minimum Dry Time
	Pre-vacuum	3 mins	132°C /270°F	20 mins
	Do not overload the sterilizer			
Storage	Keep device in sterilized packaging until ready for use. Keep out of sunlight and moisture			
Additional Information	If sterilization packaging is compromised, repeat sterilization process This procedure was validated by an independent and accredited laboratory			

