



# Seek and Find

## Sable™ Seek™ and Seek™


### CARIES INDICATORS

Be certain you've identified all demineralized dentin. With highly visible colors and mess-free syringe delivery, Sable Seek and Seek caries indicators make finding caries a simple process of seek and find.



# Sable™ Seek™ and Seek™

- Highly visible, they clearly distinguish demineralized dentin for efficient, confidence-inspiring removal
- Precise, mess-free syringe delivery
- Patient-friendly dyes
- Green Sable Seek caries indicator helps visualization of decay in deep caries cases to help avoid pulp exposures
- Both Seek and Sable Seek caries indicators can provide a fast, effective way to locate root canal orifices

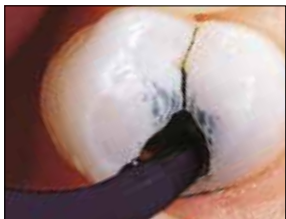

1. Apply Sable Seek indicator with Black Mini Brush tip.



2. Rinse with air/water and suction. Carious dentin is easily identified.



3. Remove green-black color (cariou dentin) with slow-speed round bur or excavator. To control overexcavating near the pulp, remove final portion of caries with hand excavator.



4. Reapply. Rinse and verify complete caries removal.



**233 - Sable Seek Syringe Kit**  
4 x 1.2 ml syringes  
20 x Black Mini Brush tips



**209 - Seek Syringe Kit**  
4 x 1.2 ml syringes  
20 x Black Mini Brush tips



**234 - Sable Seek Syringe 4pk**  
**1805 - Sable Seek Syringe 20pk**  
1.2 ml syringes



**210 - Seek Syringe 4pk**  
**1804 - Seek Syringe 20pk**  
1.2 ml syringes

