

Seek and Find

Sable™ Seek™ and Seek™


CARIES INDICATORS

Be certain you've identified all demineralized dentin. With highly visible colors and mess-free syringe delivery, Sable Seek and Seek caries indicators make finding caries a simple process of seek and find.




Sable™ Seek™ and Seek™


- Highly visible, they clearly distinguish demineralized dentin for efficient, confidence-inspiring removal
- Precise, mess-free syringe delivery
- Patient-friendly dyes
- Green Sable Seek caries indicator helps visualization of decay in deep caries cases to help avoid pulp exposures
- Both Seek and Sable Seek caries indicators can provide a fast, effective way to locate root canal orifices

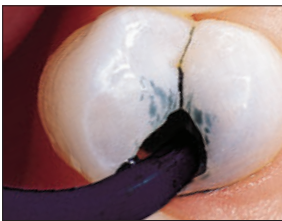

1. Apply Sable Seek indicator with Black Mini Brush tip.

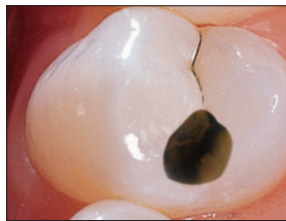

2. Rinse with air/water and suction. Carious dentin is easily identified.

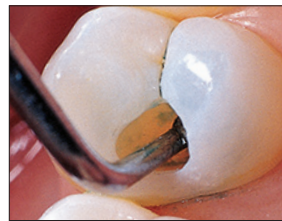

3. Remove green-black color (carious dentin) with slow-speed round bur or excavator. To control overexcavating near the pulp, remove final portion of caries with hand excavator.



4. Reapply. Rinse and verify complete caries removal.



233 - Sable Seek Syringe Kit
4 x 1.2 ml syringes
20 x Black Mini Brush tips



209 - Seek Syringe Kit
4 x 1.2 ml syringes
20 x Black Mini Brush tips



234 - Sable Seek Syringe 4pk
1805 - Sable Seek Syringe 20pk
1.2 ml syringes



210 - Seek Syringe 4pk
1804 - Seek Syringe 20pk
1.2 ml syringes