# TriAway™

**Syringe Adapter/DE-Spritzen-Adapter/FR-Adaptateur pour Seringue/  
NL-Adapter voor driefunctiespuit/IT-Adattatore per siringhe/ES-Adaptador  
de Jeringa/PT-Adaptador de Seringa/SV-Sprutadapter/DA-Adapter til  
sprøjte/FI-Ruiskuadapteri/EL-Προσαρμογέας αεροσύριγγας**

Blue Max® Impression Tip

Black Mini® Tip

White Mac® Tip

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FINNISH

GREEK

U.S. federal law restricts this device to sale by or on the order of a dentist.

Keep out of reach of children.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200.

EN- See Instructions  
DE- Siehe Anleitung  
FR- Cf. instructions  
NL- Zie gebruiksaanwijzing  
IT- Vedere le istruzioni  
ES- Vea las Instrucciones  
PT- Veja as instruções  
SV- Se anvisningar  
DA- Se brugsanvisningen  
FI- Lue käyttöohjeet  
EL- Δείτε τις οδηγίες

EN- Recycle  
DE- Recyceln  
FR- Recycler  
NL- Recyclen  
IT- Riciclare  
ES- Reciclar  
PT- Reciclar  
SV- Återvinna  
DA- Genbruges  
FI- Kierrätä  
EL- Ανακυκλώστε το

EN- Use by date  
DE- Verfallsdatum  
FR- Date de péremption  
NL- Uiterste gebruiksdatum  
IT- Utilizzare entro  
ES- Usar antes de  
PT- Usar até a data  
SV- Använd före datum  
DA- Udløbsdato  
FI- Viimeinen käyttöajankohta  
EL- Ημερομηνία λήξης

EN- Lot Number  
DE- Chargennummer  
FR- Numéro du lot  
NL- Batchnummer  
IT- Numero del lotto  
ES- Número de Lote  
PT- Número do lote  
SV- Satsnummer  
DA- Lotnummer  
FI- Eränumero  
EL- Αριθμός παρτίδας

EN- For professional use only  
DE- Nur zur Verwendung durch Sachkundige  
FR- Destiné à un usage professionnel uniquement  
NL- Alleen voor professioneel gebruik  
IT- Esclusivamente ad uso professionale  
ES- Sólo para uso profesional  
PT- Apenas para uso profissional  
SV- Enbart för yrkesmässig användning  
DA- Kun til professionel brug  
FI- Vain ammattikäyttöön  
EL- Για επαγγελματική χρήση μόνο

EN- Catalog Number  
DE- Katalognummer  
FR- Numéro de référence  
NL- Catalogusnummer  
IT- Numero di catalogo  
ES- Número de catálogo  
PT- Número do catálogo  
SV- Katalognummer  
DA- Katalognummer  
FI- Luettelonumero  
EL- Αριθμός καταλόγου

EN- Do not re-use to avoid cross contamination  
DE- Um Kreuzkontaminierung zu vermeiden, nicht erneut benutzen  
FR- Ne pas réutiliser pour éviter la contamination croisée  
NL- Niet opnieuw gebruiken om kruisbesmetting te voorkomen  
IT- Non riutilizzare per evitare la contaminazione crociata  
ES- No reutilice para evitar la contaminación cruzada  
PT- Não reutilize para evitar a contaminação cruzada.  
SV- För undvikande av korskontamination, återanvänd inte  
DA- Må ikke genbruges for at undgå krydskontaminering  
FI- Älä käytä uudelleen ristikontaminaation vältämiseksi.  
EL- Μην το επαναχρησιμοποιείτε για να αποφύγετε τη διασταυρούμενη μόλυνση

## TriAway™ Syringe Adapter

### Description:

The TriAway™ syringe is a disposable irrigator/dryer to be used with Ultradent® delivery tips on three-way syringes. It will fit most three-way syringe tips with an outside diameter of 0.1523"/3.86mm.

### Indications:

Use the TriAway syringe for controlled application of water and/or air to depths of tiny cavity preparations, whether they are minimally invasive operative-type preparations or endodontic preparations. Removal of water or excess bonding agents from even acute corners and line angles of larger preparations occurs with precision and superior predictability and quality.

### Procedure:

1. Slip TriAway snugly over three-way syringe tip.
2. Attach Ultradent syringe tip of choice firmly to end.
3. Experiment with air/water, air or water flow on inert surface prior to use in mouth.

### Precautions:

1. Do not force compressed air or water into endodontic canals or surgical sites.
2. Tips with a smaller diameter than a Max Tip (0.055" ID) should only be used with air or water - not both at the same time.

Use of air/water with smaller diameter tips will not allow flow of both and may lead to backflow.

3. To prevent unwanted dislodgement, verify that all components are firmly attached.
4. Use light to moderate pressure with smaller diameter tips. Pneumatization of tissues may occur if tips are directed into sulcular tissues or surgical sites.
5. Insert TriAway to end of three-way syringe tip.
6. Apply petroleum jelly to three-way syringe tip before inserting TriAway.
7. Do not autoclave the TriAway syringe.

Fig. 1 Black Mini® or Blue Max® Impression tips are recommended for use with the TriAway syringe. The Blue Max or White Mac® (shown here) provide a more predictable air water flow although access into micro preparations may be limited.

## TriAway™ Syringe Adapter

Manufactured by:  
Ultradent Products Inc  
505 West 10200 South  
South Jordan, Utah 84095 USA  
Made in USA

Ultradent Products GmbH  
Am Westhofer Berg 30  
51149 Cologne Germany

**ULTRADENT**  
PRODUCTS, INC.