

UltraTemp™

Luting/Filling Material



The right amount of

STRENGTH

for the right amount of

TIME!



UltraTemp™ polycarboxylate, non-eugenol temporary luting/filling material is your solution for walking bleach cases, temporary crowns, bridges, inlays, onlays, and small temporary fillings. It features a non-eugenol formula that will not interfere with resin bonding and with its convenient mixing tip delivery, UltraTemp luting/filling material provides a highly convenient, time-saving application method that delivers an even mix of cement for reliable strength. Once set, UltraTemp luting/filling material provides optimal sealing capabilities. While it is strong enough to withstand normal biting and chewing forces, it will still easily remove when you want it to.

UltraTemp™

UltraTemp™ POLYCARBOXYLATE, NON-EUGENOL TEMPORARY LUTING/FILLING MATERIAL

A hydrophilic, polycarboxylate chemistry ensures low irritation to pulp and a quality seal. It can be easily removed with water prior to setting. UltraTemp luting/filling material is suggested for routine 1–2 week temporization of custom-fabricated provisionals or standard preformed provisionals. It can also be used to seal the access opening of walking bleach cases.

Temporary Provisional Luting



Prior to complete set, remove excess UltraTemp luting/filling material easily with a moist cotton swab or gauze. Following subgingival set of 1–2 minutes, remove quickly with explorer.



Upon provisional removal, shown here two weeks post-op, cement clings to both provisional and preparation. This is one indicator of a quality sealing cement.



Flake off residual cement with blunt hand instrument.



Use an abrasive-like ConsepSis™ Scrub CHX antibacterial slurry with a rubber cup or STARbrush™ intercoronal brush to remove residual cement.

Walking Bleach Case



1. After following the instructions to place Opalescence™ Endo 35% hydrogen peroxide non-vital “walking bleach” to the tooth, deliver UltraTemp luting/filling material into the chamber with an Ultradent Intraoral tip.



2. Easily wipe away excess with a wet cotton ball or gauze before it sets.



3. Finished. Repeat every 1–5 days until desired results are achieved.

Courtesy of Dr. Carlos Ramos.



REFRIGERATE
5916 - UltraTemp Regular Kit 2- to 3-Minute Set Time
1 x 5 ml UltraTemp syringe
20 x Mixing tips



1. realityesthetics.com
© 2020 Ultradent Products, Inc. All rights reserved.
68492AR14 082719

800.552.5512 | ULTRADENT.COM

