DeOx™ <div>Oxygen Barrier Solution</div>	EN
--	----

Description:
DeOx barrier solution is a clear, viscous, glycerin-based solution designed to prevent the formation of an oxygen inhibition layer on the surface of resin materials when they polymerize.

Indications:
To prevent a negative void or contour in areas where composite or luting cement is contoured so close to the final dimension that removal of the layer would leave a void. Allows removal of luting resins closer to the final contour prior to curing, thus eliminating the need to address a large excess of cured composite or luting cement.

Procedure:

- When placing composite restorations, complete the restoration but do not cure. For oxygen inhibition control when luting, follow manufacturer’s instructions but do not cure.
- Remove the Luer Lock cap from the syringe of DeOx barrier solution and securely attach the disposable tip onto the syringe.
- Apply DeOx solution. Cover all exposed resin. In the case of an indirect restoration or full composite direct resin restoration, be sure to cover the margins. Take care to fill all embrasures (i.e., gingival, buccal, and lingual).
- Allow exposed resin to completely polymerize (light or chemical cure) according to manufacturer’s instructions.
- Remove excess DeOx solution with air/water spray.

Precautions and Warnings

- For professional use only.
- Always verify material flow prior to applying intraorally. If resistance is met, replace tip and re-check.
- Carefully read and understand all instructions before using.
- Do not use on patients with a known sensitivity to ingredients. If allergic reaction, dermatitis, or rash develops, consult a physician.
- Do not use after expiration date noted on containers.
- Tips are disposable. To avoid cross-contamination, do not re-use tips.
- Properly dispose of tips and empty syringes according to local government regulations.
- All intra-oral syringes must be used with disposable syringe covers and wiped with disinfectant between uses. If these measuress are not taken, the syringes should be considered single-use.

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DeOx™ <div>Flammability</div>	Reactivity
<div><div><div><div><div></div></div></div><div><div><div></div></div><div><div>1</div></div></div><div><div><div></div></div><div><div>1</div></div></div><div><div><div></div></div><div><div>0</div></div></div></div></div>	
HAZARD RATINGS <div> <ul style="list-style-type: none">4 = Severe 3 = Serious 2 = Moderate 1 = Slight 0 = Minimal </div>	
For product MSDS please see our website: www.ultradent.com	

U.S. federal law restricts the sale of this device by or on the order of a dentist.
Keep out of reach of children.
For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512.Outside U.S. call (801) 572-4200 or visit www.ultradent.com.

For product SDS please see our website: www.ultradent.com
1-800-552-5512; 801-572-4200

DeOx™ <div>Oxygen Barrier Solution</div>
Manufactured by: <div> <div><div><div></div></div></div> <div> <div><div></div></div> <div><div>Ultradent Products Inc</div></div> <div><div>505 West Ultradent Drive (10200 South) South Jordan, Utah 84095 USA</div></div> <div><div>Made in the USA from U.S. and globally sourced materials</div></div> </div> </div>