

Opalescence
● ● ● go®

A FLAWLESS FINISH



Opalescence Go whitening features the UltraFit™ tray, which adapts to the smile of any patient—even those wearing orthodontic appliances. The unique tray material easily adapts around brackets for a comfortable and effective whitening experience. Opalescence Go whitening is the perfect option for whitening smiles with braces and giving every patient an extra measure of confidence at every stage of their orthodontic treatment.



ULTRADENT
PRODUCTS, INC.

800.552.5512 ULTRADENT.COM

Opalescence go[®]



Features of Opalescence Go Whitening

- Unique tray material offers an extremely comfortable fit and easily conforms to any patient's individual smile
- Molar-to-molar coverage ensures the gel comes in contact with more posterior teeth
- UltraFit™ tray's superior adaptation ensures the maximum amount of gel stays in contact with the teeth during whitening
- Available in 10% hydrogen peroxide (with wear times from 30–60 minutes)
- Delicious Mint flavor
- Convenient prefilled trays can be worn right out of the package
- Optimal gel quantity allows easy cleanup after whitening
- Sleek packaging provides superior stability, shelf life, and branding

4635 Mint 10% 6pk (6 Patient Kits)
Each kit contains 10 upper/10 lower trays

4645 Mint 10% 12pk (12 Mini Kits)
Each mini kit contains 4 upper/4 lower trays

4654 Mint 10% Sample Dispenser Kit
*1 x Sample dispenser display
20 x upper trays
20 x lower trays
25 x Patient instructions*

Instructions



1. Remove from package.



2. Center tray on arch.



3. Gently suck down or swallow.



4. Remove outer tray.



5. Suck down or swallow again.



6. Wear for 30 to 60 minutes.

800.552.5512 ULTRADENT.COM



© 2018 Ultradent Products, Inc. All rights reserved.
1001724.3 120517

ULTRADENT
PRODUCTS, INC.

Improving Oral Health Globally