

# Powerful, professional whitening to go!



**Opalescence Go** whitening features ready-to-go convenience with the UltraFit™ tray for an even more adaptable and comfortable whitening experience.

- Unique tray material offers a remarkably comfortable fit and easily conforms to any patient's smile
- Molar-to-molar coverage ensures the gel comes in contact with more posterior teeth
- UltraFit tray's superior adaptation ensures the maximum amount of gel stays in contact with the teeth during whitening
- Available in 10% hydrogen peroxide (with wear times from 30–60 minutes) and 15% hydrogen peroxide (with wear times from 15–20 minutes)



Opalescence  
● ● ● go™



**ULTRADENT**  
PRODUCTS, INC.

800.552.5512 [ULTRADENT.COM](http://ULTRADENT.COM)



### Additional features of Opalescence Go whitening

- Mint and Melon flavors
- Convenient prefilled trays can be worn right out of the package
- Optimal gel quantity allows easy cleanup after whitening
- Every patient kit includes a 1 oz. tube of Opalescence™ Whitening toothpaste

#### Case of 6 Patient Kits

(10 upper/lower trays and a 1-ounce tube of toothpaste per kit)

- 4635-US 10% Mint 6pk
- 4636-US 10% Melon 6pk
- 4638-US 15% Mint 6pk

#### Case of 12 Mini Kits

(4 upper/lower per kit)

- 4645-US 10% Mint 12 Kits
- 4646-US 10% Melon 12 Kits
- 4648-US 15% Mint 12 Kits

#### Sample Towers

(20 upper/lower per tower)

- 4893-US 10% Mint Sample Tower
- 4890-US 10% Melon Sample Tower
- 4894-US 15% Mint Sample Tower

Refrigeration is recommended for all Opalescence Go whitening products.


### Instructions



1. Remove from package.




2. Center tray on arch.

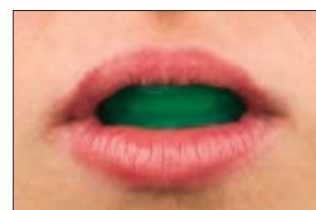

3. Gently suck down.



4. Remove outer tray.

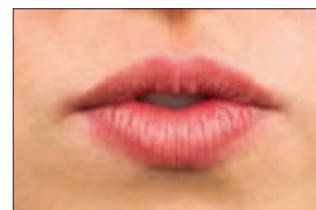

5. Suck down.



6. Wear 10% for 30 to 60 minutes, and 15% for 15 to 20 minutes.

