



Ultra-Blend[®] plus

Cavity Liner and Pulp-Capping Material



Ultra-Blend plus liner is a unique dentin-adhering liner and base material containing calcium hydroxide and calcium hydroxyapatite in a urethane dimethacrylate (UDMA) base. It's perfect for pulp capping and will not dissolve over time. Made of a light-activated, radiopaque material, Ultra-Blend plus liner is highly filled for minimal shrinkage.

- Bioactive¹
- Superior calcium release²
- Controlled, precise syringe delivery
- No mixing necessary
- Light cured when clinician is ready
- Radiopaque

1. Pameijer CH, Stanley HR, Norval G. Pulp capping with a new adhesive light cured calcium hydroxide formula [AADR abstract 757]. *J Dent Res.* 1998;77(suppl 1):200. 2. Data on file.



ULTRADENT
PRODUCTS, INC.

800.552.5512 ULTRADENT.COM

Ultra-Blend[®] plus liner application:

A bioactive base material is an essential tool used in restorative dentistry—especially in cases where the pulp is exposed. Thanks to its superior calcium release, Ultra-Blend plus liner effectively stimulates the reparative process.¹ That's one reason it's been a preferred pulp-capping material for over 15 years.




Use Ultra-Blend plus liner near pulp (pink) and for small nonhyperemic exposure. If larger exposure and/or hyperemic pulp, initiate endodontic therapy.




Apply ConsepSis[®] antibacterial solution with the plastic Blue Mini[®] Dento-Infusor[®] tip for 60 seconds with gentle scrubbing action. Gently air dry (don't rinse).



With the Black Micro[®] tip, apply Ultra-Blend plus liner to dry dentin at exposure border (or over thin pink dentin); cover exposure and light cure. Minimize dentin coverage to maximize available dentin for bonding.

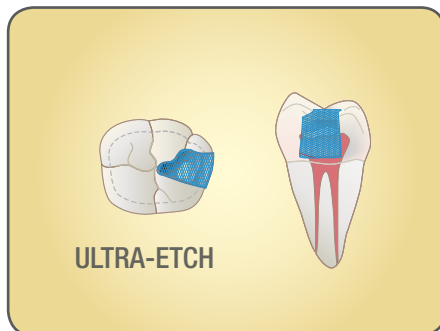

Apply Ultra-Etch[®] 35% phosphoric acid etchant solution for 20 seconds. Suction, rinse, and dry until damp.

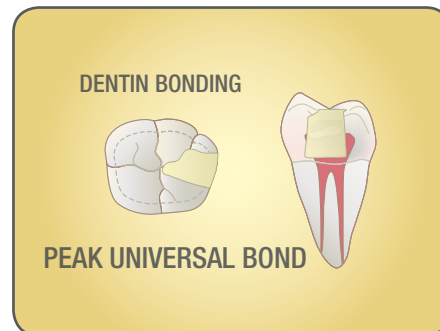

With the Inspiral[®] Brush tip, apply Peak[®] Universal Bond adhesive, rubbing and agitating into the preparation for 10 seconds. Air thin at half pressure for 2–3 seconds and light cure for 10 seconds with the VALO[®] curing light on Standard mode. Restore with a quality composite.

415 Ultra-Blend plus Kit

- 2 x 1.2 ml Ultra-Blend plus Dentin syringes
- 2 x 1.2 ml Ultra-Blend plus Opaque White syringes
- 20 x Black Micro tips
- 20 x Black Mini tips



416 Ultra-Blend plus Dentin Refill 417 Ultra-Blend plus Opaque White Refill

- 4 x 1.2 ml syringes

