| Enamelast™ Fluoride Varnish 5% Sodium Fluoride Varnish | EN |
|--|----|
|--|----|

Product Description
Enamelast™ fluoride varnish is a flavored, xylitol-sweetened, 5% sodium fluoride in a resin carrier delivered in a 1.2 ml syringe or in a 0.4 ml unit dose blister pack. For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Form(s)

- 1.2 ml delivery syringe
- SoftEZ™ tip
- 0.4 ml unit dose pack

Application Instructions

Indications for Use/Intended Purpose

Enamelast fluoride varnish is 5% sodium fluoride in a varnish carrier which produces a mechanical occlusion of the dental tubules in the treatment of tooth hypersensitivity.

Contraindications

- Do not use on patients with ulcerative gingivitis or stomatitis.
- Do not use on patients with a known sensitivity to ingredients including colophony (kolofonium) and/rosin. If allergic reaction, dermatitis, or rash develops, consult a physician.
- For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions

- Use only recommended tips.
- Tips are single use only.
- Patient, clinician, and assistant must wear protective eyewear.
- In rare instances, edematous swelling has been reported after extensive fluoride varnish treatment and dyspnea has been reported in asthmatic children. If this occurs, thoroughly brush teeth to remove the varnish film and have patient contact their medical provider.
- Some patients may experience nausea after application. If nausea persists, thoroughly brush teeth to remove the varnish film.
- Do not use more than half of the 1.2 ml syringe of fluoride varnish per patient per application.
- Do not use with other topical fluoride treatments the same day and instruct patient to refrain from using any fluoride supplements for 4 (four) days after treatment.
- Do not remove cap from the syringe or foil lid from the unit dose until ready to use.

Stepwise Instructions

Preparation

- Thoroughly brush teeth, rubber cup polish, or wipe teeth with gauze prior to placement.
- For syringe:
 - Securely attach recommended delivery tip and verify the flow, color and consistency of the product before applying by expressing a small amount onto a mixing pad or gauze. Discard any clear liquid that is expressed. If resistance is met, replace tip and recheck.
 - For unit dose:
 - Open blister pack by peeling back and removing the foil lid. Using the disposable application brush, lightly stir the varnish.

Use

- Dry area to be treated.
- Apply a thin, smooth layer of Enamelast fluoride varnish to as many surfaces as possible.

NOTE:

- Patients should avoid brushing, flossing, for 4–6 hours after treatment. If possible, wait until the next morning.
- Patients should avoid hot foods or drinks as well as products containing alcohol (e.g., beverages, oral rinses, etc.) for 4–6 hours after treatment.
- Enamelast fluoride varnish will begin to flake off with each brushing.
- For maximum benefit, Enamelast varnish can be reapplied every 3–6 months.

Processing

Enamelast varnish is a contamination-free, vinyl syringe with an intermediate level disinfectant between uses and a use a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be stored as single use.

Storage and Dispose

Dispose of waste according to local rules, guidelines, and regulations.

Miscellaneous Information

Report any serious incident to the manufacturer and the competent authority. www.ultradent.com / 1-800-552-5512 / 801-572-4200

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