



The strength is in the preparation.

Ultradent™ Porcelain Etch, a buffered 9% hydrofluoric acid, produces an optimal surface texture on silica-based ceramics after a 90-second application. Once rinsed and followed by a 60-second application of Ultradent™ Silane coupling agent, the highest porcelain-to-resin bond strengths can be achieved.¹

- Easy to control and place
- Will not stain composite or resin cement
- Yields highest porcelain-to-resin bond strengths¹


Porcelain Etch and Silane

Ultradent Porcelain Etch is a buffered, 9% hydrofluoric acid and is ideal for intraoral and extraoral porcelain etching. It offers high visibility and is viscous enough to stay where it's placed.



For the best bond strengths, follow porcelain etching with an application of Ultradent Silane, a single-component coupling agent.

- Syringe/tip delivery eases placement
- 90-second etch, 60-second silane
- High porcelain-to-resin bond strengths
- Predictable, stain-free chemistries
- Stable formulation with long shelf life

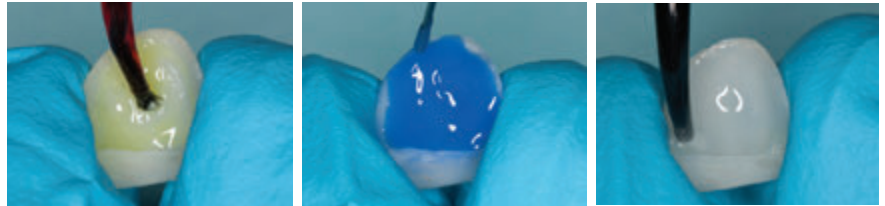

Etch ceramic bonding surface with Porcelain Etch etchant for 90 seconds, rinse, and dry.

CLINICAL TIP: Apply Ultra-Etch™ etchant for five seconds to remove porcelain salts and debris formed by hydrofluoric acid etching.

Apply a puddle coat of Silane coupling agent to the inside surface of the prosthesis for 60 seconds, dry, and set aside. Do not rinse. Prosthesis now ready for luting/cementing.

For best results, use Porcelain Etch etchant with the Inspiral™ Brush tip and Silane coupling agent with the Black Mini™ Brush tip.



405 - Porcelain Etch Syringe Kit
 2 x 1.2 ml Porcelain Etch syringes
 2 x 1.2 ml Silane syringes
 20 x Black Mini Brush tips
 20 x Inspiral Brush tips



406 - Porcelain Etch Syringe 2pk
 1.2 ml syringes



410 - Silane Syringe 2pk
 1.2 ml syringes


1108 - Ultradent Porcelain Repair Syringe Kit

- 1 x 1.2 ml PermaFlo Dentin Opaquer syringe
- 1 x 1.2 ml EtchArrest syringe
- 1 x 1.2 ml OpalDam syringe
- 1 x 1.2 ml Peak Universal Bond syringe
- 1 x 1.2 ml Porcelain Etch syringe
- 1 x 1.2 ml Ultradent Silane syringe
- 20 x Black Mini Brush tips
- 20 x Black Micro tips
- 20 x Micro 20 ga tips
- 20 x Inspiral Brush tips



The repair of a fractured porcelain veneer begins with Porcelain Etch acid. (Photo courtesy of Shannon Pace, CDA, CDD)

Ultradent Porcelain Etch etchant and Silane agent may also be used on lithium disilicate (IPS e.max®) restorations

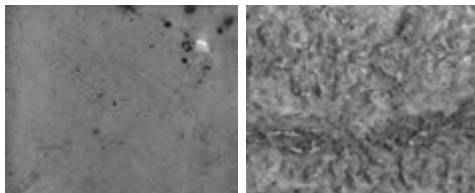

Lithium disilicate surface before being etched²

After 90-second application of Porcelain Etch etchant

2. Data on File

Product	Instructed Etching Time	Coupling Agent	Shear Bond Strength
Ultradent Porcelain Etch (9% hydrofluoric acid)	90 seconds	Silane 60 Seconds	44.30 MPa ³
IPS Ceramic Etching Gel (5% hydrofluoric acid)	20 seconds	Monobond®* Plus 60 seconds	28.63 MPa ³

3. Data on File
 *Registered trademark of a company other than Ultradent