Vit-escence™

Manufactured by:
Ultradent Products Inc
505 West Ultradent Drive (10200 South)
South Jordan, Utah 84095 USA

Manufactured in the USA from
globally sourced materials
www.ultradent.com

ULTRADENT
PRODUCTS, INC.

For immediate reorder and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call Toll Free 1-800-552-5512. Outside U.S. call (801) 572-4200 or visit www.ultradent.com.

For SDS and additional information about using Vit-escence™ and other related products, please go to www.ultradent.com

RxOnly

EN - For professional use only
ES - Solo para uso profesional

Vit-escence™ Universal Composite

Product Description:

Vit-escence™ composite is a light-cured, microhybrid, Bis-GMA esthetic composite restorative material. It is 75% filled by weight, 52% filled by volume with 0.7 µm average particle size.

Dentin Shades: fluorescent properties to replicate natural dentin. Available in 18 dentin shades.

Enamel Shades: opalescent properties to replicate enamel. Available in 16 enamel shades including pearl, translucent, and opaque shades

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Form(s)

- 2.5 g Syringe
- 0.3 g Single use capsule

Composition

> 2.5% - ≤ 10% tetramethylene dimethacrylate

Indications for Use/Intended Purpose

Vit-escence composite is used for direct and indirect esthetic restorations in anterior and posterior regions.

Contraindications

- Do not use on patients with known sensitivity to methacrylates or other resins. If dermatitis, rash, or other allergic reaction is observed, rinse exposed area thoroughly and have patient consult their physician.
- For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician

Warnings and Precautions

- Resins can be sensitizing. Do not use on patients with a known sensitivity to acrylates or other resins. If allergic reaction, dermatitis, or rash develops, have patient consult their physician.
- Syringes are not for intra-oral use.
- To avoid cross-contamination, use clean gloves and a clean instrument.
- Redirect overhead light to prevent premature polymerization.
- To optimize bond to each succeeding layer, do not remove oxygen inhibition layer between incremental layers of composite.
- Ensure composite is totally polymerized between layers.
- Patient and dental professionals must wear protective eyewear with side shields.
- If the product is ingested, drink large amounts of water and contact a physician. If the product is aspirated, seek immediate medical attention.

Stepwise Instructions

Preparation

1) Apply dentin bonding agent to preparation according to manufacturer's instructions.

Use - Direct Restorations

- 1) Open cap and twist handle (stem) clockwise to express desired amount of composite. Resistance will lessen with each twist.
- 2) Place composite in incremental layers applying the first layer approximately 0.5–1.5 mm thick and cure. Follow with up to 2 mm increments; curing after each increment. For curing lights with an intensity of ≥ 800 mW/cm² in the wavelength region of 385–515 nm, cure each layer for 10 seconds. A final cure of 20 seconds is recommended.
- 3) After final increment is placed, sculpt and light cure.
- 4) Shape, finish, and adjust occlusion using preferred method.
- 5) Immediately replace cap after dispensing composite to prevent polymerization from ambient light.

Use - Indirect Restorations

- 1) Make impression of prepared tooth and opposing dentition.
- 2) Prepare master cast and articulate opposing cast using preferred technique.
- 3) Place composite in incremental layers applying the first layer approximately 0.5–1.5 mm thick and cure. Follow with up to 2 mm increments; curing after each increment. For curing lights with an intensity of ≥ 800 mW/cm² in the wavelength region of 385–515 nm, cure each layer for 10 seconds.
- 4) Use final increments to create proximal contours and ensure marginal adaptation, a final cure of 20 seconds is recommended.
- 5) Remove restoration. Clean off residual die stone and trim marginal flash. Additions can be made by roughening restoration surface, applying adhesive, and placing composite as needed.
- 6) Sandblast internal aspect of restoration with 50 µm alumina abrasive.
- 7) Seat restoration onto master cast, evaluate fit, occlusion, and contours. Shape, finish, and adjust occlusion using preferred method.
- 8) Bond restoration intraorally per adhesive manufacturer's instructions.
- 9) Following cementation, make necessary occlusal adjustment and polish adjusted surfaces.

Use - Singles (capsule), Direct Restoration

- 1) Keep capsule caps on until use.
- 2) Snap capsule firmly into composite gun before trying to express.
- 3) Unit dose capsules are designed for single patient use only. Do not reuse to avoid the risk of cross contamination. Discard capsule after use.
- 4) Place composite in incremental layers applying the first layer approximately 0.5–1.5 mm thick and cure. Follow with up to 2 mm increments; curing after each increment. For curing lights with an intensity of ≥ 800 mW/cm² in the wavelength region of 385–515 nm, cure each layer for 10 seconds. A final cure of 20 seconds is recommended.
- 5) After final increment is placed, sculpt and light cure.
- 6) Shape, finish, and adjust occlusion using preferred method.

Storage and Disposal

Store according to outer package label.

Dispose of waste according to local rules, guidelines, and regulations.

Miscellaneous Information

Report any serious incident to the manufacturer and the competent authority.
www.ultradent.com / 1-800-552-5512 / 801-572-4200

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