


# TISSUE MANAGEMENT Decision Tree



We've designed our tissue management products to work together. This means clinicians can customize treatment, with interchangeable formulas and instruments, depending on the case. Whether you're taking impressions, completing a Class V restoration, or placing a crown, there's a product that is good, better, and best.

The final decision is naturally up to the doctor's preference, but here are our recommended uses for our tissue management line.

# TISSUE MANAGEMENT Decision Tree

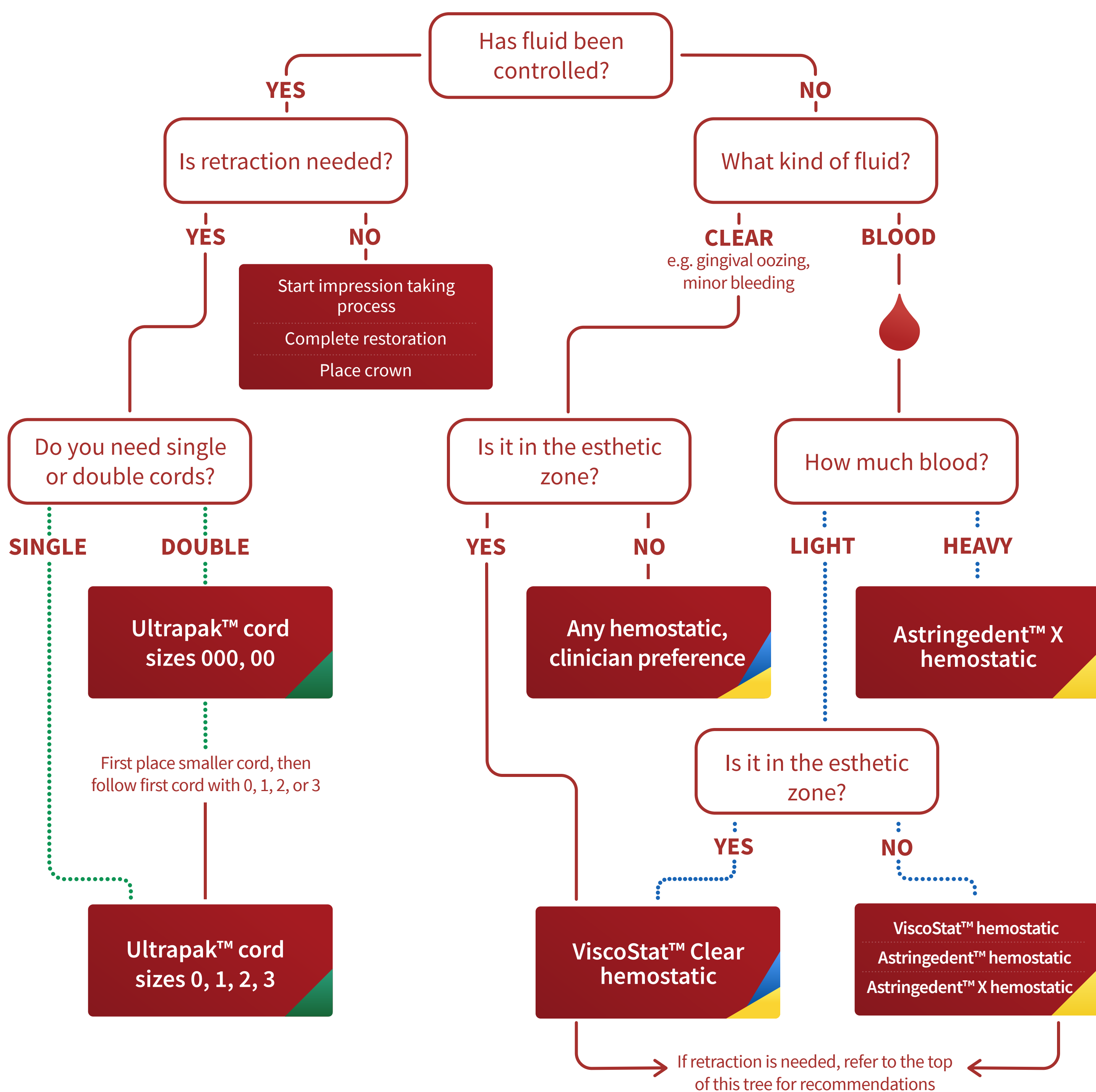

## KEY

### ..... Burnishing Technique Indicator

Be sure to scrub firmly into the sulcus. Firm pressure gets the best efficacy from the hemostatic agent chosen.

### ..... Cord Soak Technique Indicator

Soak the first cord in a hemostatic agent.

### ▲ Fischer's Ultrapak™ Packers

Specially designed to ease the packing of Ultrapak knitted cord. The thin edge helps press into the cord and serrated edge reduces the risk of slipping around the gingival attachment.

### ▲ Blue Mini™ Dento-Infusor™ Tip

- Used when fragile healing tissue or sensitivity is present
- If only control of sulcular fluid is required, the softer tip end of the plastic blue DI may be gentler

### ▲ Metal Dento-Infusor™ Tip

- Effectively places hemostatic agents. It's best for bleeding, especially heavy bleeding, and removes superficial coagulum.
- Has a blunt, bent cannula with a padded "scrub brush" for infusing and cleaning the cut sulcus