

# amelogen™ plus

composite restorative material



- Intuitive shade system
- Superior polish
- Economically priced


Amelogen Plus composite offers a balance between strength and esthetics without overcomplicating the procedure. With 15 shades to choose from, you'll be able to easily replicate natural tooth color.



Shades offered in both syringe and single delivery.

## Intuitive Shade System

Amelogen Plus composite has been created with an intuitive shade system that allows you to choose a one-, two-, or multiple-shade layering technique. The composite material is available in 15 shades.

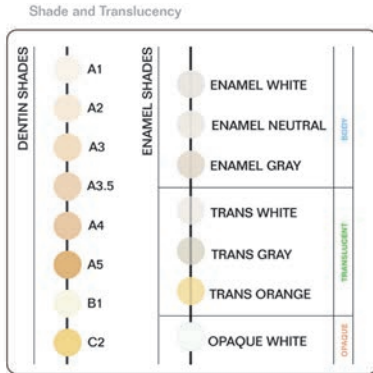

Beautiful restorations using only 2 shades.



BEFORE AFTER  
Everything you need to create beautiful restorations every time. Understandable. User-friendly. Economical.

Dr. Michael J. Melkers  
Dr. Jaleena Jessop

## Multiple-shade Placement

This technique is recommended for large restorations and direct veneers to re-create the essence of natural tooth structure.

1. Choose the dentin shade by matching to exposed dentin or the cervical 1/3 of the tooth.
2. Choose the value (brightness) of the overall tooth.
3. Deliver etch and bonding agent to fully prepared tooth.
4. Place the first 0.5–1 mm layer of dentin composite. Continue building up additional layers in 2 mm increments, replicating the natural anatomy of the surrounding dentin tooth structure. Cure 10 seconds between each layer.
5. Add characterizations by using an enamel shade and/or Opaque White.
6. Add translucency to the incisal edge to replicate the natural halo, staining, or mamelons.
7. Place enamel composite layer, building up in 1–2 mm increments, replicating the natural anatomy of the surrounding enamel tooth structure. Cure 10 seconds between each layer.
8. Shape, finish, polish, and seal.

## Ordering Information


### Amelogen Plus 2.5 g Syringes 1pk

Dentin	1pk	Enamel	1pk
A1	9030	Opaque White™	9037
A2	9031	Enamel White™	9038
A3	9032	Enamel Neutral™	9039
A3.5	9028	Enamel Gray™	9040
A4	9033	Trans White™	9041
A5	9034	Trans Gray™	9042
B1	9035	Trans Orange™	9043
C2	9036		



### Amelogen Plus 0.3 g Singles 10pk

Dentin	10pk	Enamel	10pk
A1	8010	Opaque White™	8021
A2	8011	Enamel White™	8022
A3	8013	Enamel Neutral™	8023
A3.5	8024	Trans White™	8016
A4	8014	Trans Gray™	8018
A5	8015	Trans Orange™	8019
B1	8017	Super Light™	8026
C2	8020		

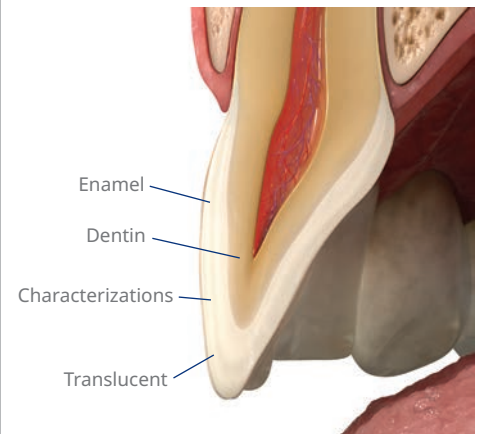

SCAN TO VIEW PRICING

