



Sensitive?
Not anymore

UltraEZ™
Desensitizing Gel



Now
Featuring The
UltraFit™
Tray

Features of UltraEZ desensitizing gel

- Provides fast, easy treatment for sensitive smiles
- Prefilled UltraFit™ tray adapts to any smile for a comfortable fit
- Also available in syringes for use with custom trays
- Quickly treats sensitivity caused by tooth whitening, abrasion, root exposure, and thermal or chemical changes
- Can be used before and after whitening treatments to dramatically reduce sensitivity



Before wearing UltraFit tray in the mouth



UltraFit tray after just 10 minutes in the mouth



- 1008 - UltraEZ Syringe 4pk
- 1007 - UltraEZ Econo Syringe 20pk
1.2 ml syringes



- 5743 - UltraEZ Mini Tray Combo 4pk
4 x Each upper/lower trays



Featuring the UltraFit™ tray

- 5721 - UltraEZ Tray Combo 10pk
10 x Each upper/lower trays



Featuring the UltraFit™ tray

Instructions



1. Remove product from packaging.
"U" - Upper tray
"L" - Lower tray



2. Position upper tray on teeth.



3. Bite firmly, then suck on tray for 2 seconds.



4. Remove colored outer tray, leaving the white inner tray on teeth. Repeat the process for the lower tray.



5. Wear for 15 minutes to 1 hour per day or as directed by your dentist. After indicated wear time, remove trays and brush teeth.

