



Opalescence  
● ● ● go™

Whiter teeth  
even with  
**braces!**

Opalescence Go whitening features the UltraFit™ tray, which adapts to the smile of any patient—even those wearing orthodontic appliances. The unique tray material easily adapts around brackets for a comfortable and effective whitening experience. Opalescence Go whitening is the perfect option for whitening smiles with braces and giving every patient an extra measure of confidence at every stage of their orthodontic treatment.



**ULTRADENT**  
PRODUCTS, INC.



WHITEN

800.552.5512 [ULTRADENT.COM](http://ULTRADENT.COM)

# Opalescence



## Features of Opalescence Go Whitening

- Unique tray material offers an extremely comfortable fit and easily conforms to any patient's individual smile
- Molar-to-molar coverage ensures the gel comes in contact with more posterior teeth
- UltraFit™ tray's superior adaptation ensures the maximum amount of gel stays in contact with the teeth during whitening
- Mint and Melon flavors
- Convenient prefilled trays can be worn right out of the package
- Available in 10% and 15% hydrogen peroxide (10% wear times from 30–60 minutes, 15% wear times from 15–20 minutes)
- Optimal gel quantity allows easy cleanup after whitening
- Every patient kit includes a 1-ounce tube of Opalescence™ Whitening Toothpaste

### Case of 6 Patient Kits

(10 upper/lower trays and a 1-ounce tube of toothpaste per kit)

- 4635-US 10% Mint 6 Kits
- 4636-US 10% Melon 6 Kits
- 4638-US 15% Mint 6 Kits

### Case of 12 Mini Kits

(4 upper/lower per kit)

- 4645-US 10% Mint 12 Kits
- 4646-US 10% Melon 12 Kits
- 4648-US 15% Mint 12 Kits

### Sample Towers

(20 upper/lower per tower)

- 4893-US 10% Mint Sample Tower
- 4890-US 10% Melon Sample Tower
- 4894-US 15% Mint Sample Tower

Refrigeration is recommended for all Opalescence Go whitening products.

## Instructions



1. Remove from package.



2. Center tray on arch.



3. Gently suck down.



4. Remove outer tray.



5. Suck down.



6. Wear 10% for 30 to 60 minutes, and 15% for 15 to 20 minutes.

800.552.5512 [ULTRADENT.COM](http://ULTRADENT.COM)



© 2019 Ultradent Products, Inc. All rights reserved.  
1001724.5 041519

**ULTRADENT**  
PRODUCTS, INC.

Improving Oral Health Globally