




Delicious Protection, Smart Delivery

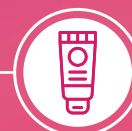

*Bulk tube
delivery*



*Economical
and efficient*



*Two delicious
flavors*



*Smooth
texture*

*gleo*TM

fluoride varnish

gleo™
fluoride varnish



Mixed Berry



Bubble Gum



Before Gleo fluoride varnish

After Gleo fluoride varnish

Confidence in Every Application

Designed for high-volume dental settings, Gleo fluoride varnish delivers 5% sodium fluoride in a non-gritty formula that goes on smooth and stays put, providing lasting benefits. The economical bulk-tube format includes enough varnish for at least 25 full treatments, reduces waste, and supports your clinic's bottom line while maximizing efficiency. With allergy-conscious ingredients* and flavors your patients will love, Gleo varnish makes it easier than ever to give your patients the fluoride protection they need.



1 Thoroughly brush teeth, rubber cup polish, or wipe teeth with gauze.



2 SHAKE WELL! Shake the tube well before every use to mix any materials that may have settled.



3 Express a small amount onto a clean surface to verify color and consistency of product. If needed, shake the tube again and express more.



4 Using a painting motion, apply a thin, smooth layer of Gleo varnish to as many dry surfaces as possible.**



5831 - Gleo Single Tube -
Mixed Berry
1 x 10 ml tube



5830 - Gleo Single Tube -
Bubble Gum
1 x 10 ml tube