Seek™ and Sable™ Seek™ Caries Indicator EN

Product Description
Seek™ caries indicator contains D&C dyes in a glycol base that stain carious dentin red. Sable™ Seek™ caries indicator contains FD&C dyes in an aqueous glycol base that stain carious dentin green.

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Forms)

- 1.2 ml delivery syringe
- Recommended delivery tip(s)
- Black Mini™ Brush

Indications for Use/Intended Purpose
Seek caries indicator is designed to aid in the complete removal of carious dentin. It is particularly useful in identifying caries in difficult to see locations and for the prevention in the over-excavating of deep caries.

Contraindications

For patients or users with allergy concerns, refer to product allergen document available at [www.ultradent.com](#). As an allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions

- Patient and dental professionals must wear protective eyewear with side shields
- Use only recommended tips
- Tips are single use only
- To avoid clogging, remove tip and re-cap with Luer Lock Caps between each use
- If the product is ingested, drink large amounts of water and contact a physician.
- If the product is aspirated, seek immediate medical attention.

- Avoid splattering. Stain is removed from acrylics using mechanical action. If dye does get on intraoral soft tissues, it will resolve on its own. If dye gets on extraoral tissue you can remove with soap and water. Use care

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