OpalDam™ / OpalDam™ Green Gingival Resin Barrier EN

Product Description
OpalDam and OpalDam Green are unique, light-effective, passively adhesive (selfing) light cured resins. Their methacrylate base is designed with adequate strength to maintain a barrier, yet removes easily from embrasures and undercuts. The contrasting green color of OpalDam Green shows its location on the gingiva and teeth for more effective coverage.

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Form(s)

- 1.2mL syringe
- Delivery tip
- Micro 20 ga Tip
- Black Mini™ Tip

Indications for Use/Intended Purpose

OpalDam and OpalDam Green are unique, light-effective, passively adhesive (selfing) light cured resins. Their methacrylate base is designed with adequate strength to maintain a barrier, yet removes easily from embrasures and undercuts. The contrasting green color of OpalDam Green shows its location on the gingiva and teeth for more effective coverage.

Contraindications

Do not use OpalDam/OpalDam Green on patients with known sensitivity to resins, especially methacrylates. If dermatitis, rash or any other allergic reaction develops, wash area with lots of water and refer patient to physician.

Warnings and precautions

- Resin can be sensitizing.
- If bubbles occur at the gingival margin of the OpalDam/OpalDam Green, reapply in that area.
- To avoid cross-contamination, wipe syringe with an alcohol disinfectant between uses and use a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be treated as single-use.
- Tip use single-use only.
- When curing resins on soft tissues, use scanning motion to prevent heat irritation from curing light.

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