OpalDam™ / OpalDam™ Green Gingival Resin Barrier EN

Product Description
OpalDam and OpalDam Green are unique, light-reflective, passively adhesive (sealing) light cured resins. Their methacrylate base is designed with adequate strength to maintain a barrier yet removes easily from embrasures and undercuts. The contrasting green color of OpalDam Green shows its location on the gingiva and teeth for more effective coverage.

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Forms

- 1.2mL syringe
- Delivery tip
- Micro 20 ga Tip
- Black Mini™ Tip

Indications for Use/Intended Purpose
OpalDam and OpalDam Green are unique, light-reflective, passively adhesive (sealing) light cured resins. Their methacrylate base is designed with adequate strength to maintain a barrier yet removes easily from embrasures and undercuts. The contrasting green color of OpalDam Green shows its location on the gingiva and teeth for more effective coverage.

Contraindications
OpalDam and OpalDam Green resin barrier can be used with known sensitivity to methacrylates or other resins.

For patients or users with allergy concerns, refer to product allergen information available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician for further advice.

Warnings and Precautions

- Resins can be sensitizing. If dermatitis, rash, or any other allergic reaction develops wash area with lots of water and refer patient to their physician.
- Use caution when using OpalDam resin barrier for isolation on patients with severe mucositis or subepidermal autoimmune disease that manifests in the oral cavity. Instead use a rubber dam in place of resin barrier.
- If bubbles occur at the gingival margin of the OpalDam Green resin barrier, reapply in that area.
- To avoid cross-contamination, wipe syringe with alcohol swab, instead use a rubber dam in place of resin barrier.
- If bubbles occur at the gingival margin of the OpalDam Green resin barrier, reapply in that area.
- To avoid cross-contamination, wipe syringe with alcohol swab, instead use a rubber dam in place of resin barrier.
- Tips are single use only.
- When curing resins on soft tissues, use scanning

motion to prevent irritation from curing light.
Stepwise Instructions

- Preparation**
- Remove Luer lock cap and attach recommended adequate strength to maintain a barrier yet removes easily from embrasures and undercuts.
 - Verify flow prior to expressing intraorally. If resistance is met, replace tip and re-check.
 - Completely rinse and dry teeth and gingival tissue prior to use.

Use

- Express OpalDam/OpalDam Green resin barrier approximately 2-3 mm onto enamel. Extend resin beyond the last tooth to be treated and into open embrasures to protect interdental papilla and lingual gingival tissue. Resin buttons extending through contacts are preferred. Use caution to avoid overfilling. OpalDam/OpalDam Green resin barrier is applied with a syringe.
- When protecting a full arch, light cure the resin barrier on standard mode using a scanning motion and a curing light. The resin barrier is approximately 1 cm from the barrier and tissue area. With a dental instrument, carefully check for a full cure. If not using on full arch, scan the barrier for approximately 1-5 seconds per tooth.

- Cleaning**
- After treatment, lift OpalDam/OpalDam Green resin barrier from surface by gently sliding a dental instrument between the dam and tissue. All or large areas can be lifted off at the patient's discretion.
 - Remove OpalDam/OpalDam Green resin barrier interproximally using explorer and/or floss. Rinse mouth thoroughly.

Processing

To avoid cross-contamination, wipe syringe with alcohol swab immediately between uses and use a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be treated as single use.

Storage and Disposal
Store according to outer package label. Dispose of waste according to local, state, and federal regulations.

Miscellaneous Information
Report any serious incident to the manufacturer and the competent authority.
www.ultradent.com / 1.800.552.5512 / 801.572.4200

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