



VISCOSTAT™ CLEAR

— VISCIOUS HEMOSTATIC —



THE CLEAR Approach to Hemostasis

SPECIALLY DESIGNED
for the esthetic zone

TRANSPARENT GEL
stays on the sulcus

LEAVES NO RESIDUE
and rinses off easily

DOES NOT INTERFERE
with bonding

ACHIEVE HEMOSTASIS . . . ESTHETICALLY

This viscous yet spreadable 25% aluminum chloride gel quickly stops minor bleeding and sulcular fluid without the mess of a liquid or paste.



Subgingival preparation with bleeding sulcus.



The Metal Dento-Infusor™ tip is used to scrub hemostatic firmly against bleeding tissues. The clear gel allows easy visibility.



Pack Ultrapak™ cord into sulcus. Leave for approximately 5 minutes.



Remove cord. Follow with a firm/air/water spray and air dry.



Finished restoration 2 weeks post-op.

- **SPECIALLY DESIGNED** for the esthetic zone
- **TRANSPARENT GEL** stays on the sulcus
- **LEAVES NO RESIDUE** and rinses off easily
- **DOES NOT INTERFERE** with bonding*



*Data on file



6407 - ViscoStat Clear Dento-Infusor IndiSpense Syringe Kit
1 x 30 ml IndiSpense syringe
20 x Metal Dento-Infusor tips
20 x 1.2 ml empty syringes



6409 - ViscoStat Clear Dento-Infusor Syringe Kit
4 x 1.2 ml syringes
20 x Metal Dento-Infusor tips



6408 - ViscoStat Clear IndiSpense Syringe 1pk
30 ml syringe



6410 - ViscoStat Clear Syringe 20pk
1.2 ml syringes