



# VISCOSTAT™ CLEAR

— VISCIOUS HEMOSTATIC —



## THE CLEAR Approach to Hemostasis

SPECIALLY DESIGNED  
for the esthetic zone

TRANSPARENT GEL  
stays where it is placed

LEAVES NO RESIDUE  
and rinses off easily

DOES NOT INTERFERE  
with bonding

## ACHIEVE HEMOSTASIS . . . ESTHETICALLY

This viscous yet spreadable 25% aluminum chloride gel quickly stops minor bleeding and sulcular fluid without the mess of a liquid or paste.



Subgingival preparation and bleeding sulcus.



Scrub hemostatic firmly against bleeding tissues with Metal Dento-Infusor™ tip. The clear gel allows easy visibility and rinses away quickly.



Place soaked Ultrapak™ cord into the sulcus. Leave for 5 minutes.



Remove cord. Firm air/water spray. Air dry. If necessary, scrub hemostatic into the sulcus again. Leave 1 minute. Facilitates great control in esthetic zone with no gingival stain.



Finished restoration 2 weeks post-op.

- **SPECIALLY DESIGNED** for the esthetic zone
- **TRANSPARENT GEL** stays where it is placed
- **LEAVES NO RESIDUE** and rinses off easily
- **DOES NOT INTERFERE** with bonding\*



\*Data on file



**6407 - ViscoStat Clear Dento-Infusor IndiSpense Syringe Kit**  
1 x 30 ml IndiSpense syringe  
20 x Metal Dento-Infusor tips  
20 x 1.2 ml empty syringes



**6409 - ViscoStat Clear Dento-Infusor Syringe Kit**  
4 x 1.2 ml syringes  
20 x Metal Dento-Infusor tips



**6408 - ViscoStat Clear IndiSpense Syringe 1pk**  
30 ml syringe



**6410 - ViscoStat Clear Syringe 20pk**  
1.2 ml syringes



SCAN  
TO VIEW  
PRICING

