

VISCOSTAT™ HEMOSTATIC



3093 - ViscoStat Dento-Infusor Syringe Kit  
4 x 1.2 ml syringes  
20 x Metal Dento-Infusor tips



647 - ViscoStat Dento-Infusor IndiSpense Syringe Kit  
1 x 30 ml IndiSpense syringe  
20 x Metal Dento-Infusor tips  
20 x 1.2 ml empty syringes



3070 - ViscoStat Syringe 20pk  
1.2 ml syringes



645 - ViscoStat IndiSpense Syringe 1pk  
30 ml syringe



1278 - Hemostatic Delivery Syringe 20pk  
3096 - Hemostatic Delivery Syringe 100pk  
1.2 ml empty syringes

ASTRINGEDENT™ X HEMOSTATIC



112 - Astringedent X Bottle 1pk  
690 - Astringedent X IndiSpense Syringe 1pk  
30 ml

ASTRINGEDENT™ HEMOSTATIC



111 - Astringedent Bottle 1pk  
686 - Astringedent IndiSpense Syringe 1pk  
30 ml



2160 - Astringedent Spot Remover 1pk  
30 ml bottle

METAL DENTO-INFUSOR™ TIP

LOK-TITE™	100pk	500pk
19 ga Metal Dento-Infusor	2559	2560



BLUE MINI™ DENTO-INFUSOR™ TIP

LOK-TITE™	mm	100pk	500pk
Blue Mini Dento-Infusor	1.20	1086	1440



! Note: Do not use epinephrine preparations with ferric sulfate products (ViscoStat, Astringedent, Astringedent X hemostatics), as blue/black precipitate will occur.

FOR PROFOUND  
HEMOSTASIS



Tissue Management

Unparalleled tissue management starts with rapid, profound hemostasis. For more than 40 years, dentists have trusted the immediate hemostatic power, detailed margins, and elimination of surface bleeding and sulcular fluid provided by Ultradent's tissue management products.

Technique Guide



1 Bleeding at preparation.



2 Astringedent X hemostatic is scrubbed firmly against bleeding tissue.



3 When bleeding stops, give a firm air/water spray to test for hemostasis and remove coagulum.



4 Profound hemostasis.



ASTRINGEDENT™ X HEMOSTATIC

12.7% Iron Solution

*The preferred hemostatic for most case situations, not just difficult ones*

- Ultradent's most powerful coagulative hemostatic
- Recommended for challenging, difficult-to-stop bleeding cases



ASTRINGEDENT™ "THE CLASSIC" HEMOSTATIC

15.5% Ferric Sulfate

*Known as the "classic" hemostatic agent—Ultradent's first product*

- Eliminates sulcular fluid contamination for optimal bonding



VISCOSTAT™ HEMOSTATIC

20% Ferric Sulfate


*Viscous formula allows for superior control*

- Provides unsurpassed kindness to tissues
- Creates rapid, profound hemostasis
- Viscous solution won't run, drip, or migrate to other teeth

TEST YOUR SCRUBBING SKILLS

Are you using adequate pressure to achieve rapid, profound hemostasis?

Directions: Use your Dento-Infusor™ tip to scrub the scratch-off area.

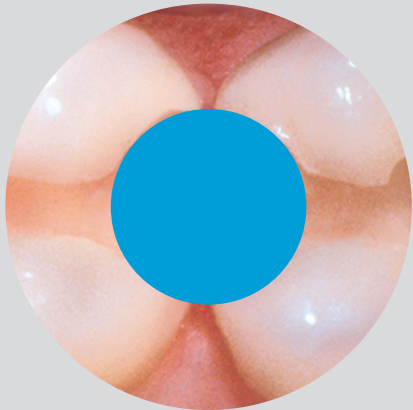

- Metal Dento-Infusor™ (MDI) has a metal, bent cannula with flared brush padded end
- Blue Mini™ Dento-Infusor™ has a softer padded end
- Designed to work with all our hemostatic agents
- Designed to be a tiny scrub brush, not only an applicator
- Facilitates infusion of hemostatics and scrubbing tissue clean of coagulum