

VISCOSTAT™ HEMOSTATIC



3093 - ViscoStat Dento-Infusor Syringe Kit
4 x 1.2 ml syringes
20 x Metal Dento-Infusor tips



647 - ViscoStat Dento-Infusor IndiSpense Syringe Kit
1 x 30 ml IndiSpense syringe
20 x Metal Dento-Infusor tips
20 x 1.2 ml empty syringes

\$



3070 - ViscoStat Syringe 20pk
1.2 ml syringes




645 - ViscoStat IndiSpense Syringe 1pk
30 ml syringe



1278 - ViscoStat Empty Syringe 20pk
3096 - ViscoStat Empty Syringe 100pk
1.2 ml empty syringes

! Note: Do not use epinephrine preparations with ferric sulfate products (ViscoStat, Astringedent, Astringedent X hemostatics), as blue/black precipitate will occur.

ASTRINGEDENT™ X HEMOSTATIC

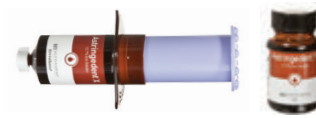

112 - Astringedent X Bottle 1pk
690 - Astringedent X IndiSpense Syringe 1pk
30 ml

ASTRINGEDENT™ HEMOSTATIC



111 - Astringedent Bottle 1pk
686 - Astringedent IndiSpense Syringe 1pk
30 ml

2160 - Astringedent Spot Remover 1pk
30 ml bottle



METAL DENTO-INFUSOR™ TIP

LOK-TITE™	100pk	500pk
19 ga Metal Dento-Infusor	2559	2560



BLUE MINI™ DENTO-INFUSOR™ TIP

LOK-TITE™	mm	100pk	500pk
Blue Mini Dento-Infusor	1.20	1086	1440



FOR PROFOUND HEMOSTASIS



Tissue Management

Unparalleled tissue management starts with rapid, profound hemostasis. For more than 40 years, dentists have trusted the immediate hemostatic power, detailed margins, and elimination of surface bleeding and sulcular fluid provided by Ultradent's tissue management products.

Technique Guide



1 Bleeding at preparation.



2 Astringedent X hemostatic is scrubbed firmly against bleeding tissue.



3 When bleeding stops, give a firm air/water spray to test for hemostasis and remove coagulum.



4 Profound hemostasis.



ASTRINGEDENT™ X HEMOSTATIC

12.7% Iron Solution

The preferred hemostatic for most case situations, not just difficult ones

- Ultradent's most powerful coagulative hemostatic
- Recommended for challenging, difficult-to-stop bleeding cases



ASTRINGEDENT™ "THE CLASSIC" HEMOSTATIC

15.5% Ferric Sulfate

Known as the "classic" hemostatic agent—Ultradent's first product

- Eliminates sulcular fluid contamination for optimal bonding



VISCOSTAT™ HEMOSTATIC

20% Ferric Sulfate


Viscous formula allows for superior control

- Provides unsurpassed kindness to tissues
- Creates rapid, profound hemostasis
- Viscous solution won't run, drip, or migrate to other teeth

TEST YOUR SCRUBBING SKILLS

Are you using adequate pressure to achieve rapid, profound hemostasis?

Directions: Use your Dento-Infusor™ tip to scrub the scratch-off area.

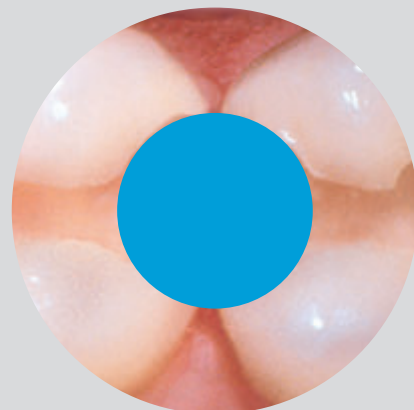

- Metal Dento-Infusor™ (MDI) has a metal, bent cannula with flared brush padded end
- Blue Mini™ Dento-Infusor™ has a softer padded end
- Designed to work with all our hemostatic agents
- Designed to be a tiny scrub brush, not only an applicator
- Facilitates infusion of hemostatics and scrubbing tissue clean of coagulum