



Ultrawave[®] XS NitiSonic Tip and Bottle Connector Kit

SAVE TIME, INCREASE EFFICIENCY

The common method of irrigation is syringe-based, which generates low fluid velocity with little interaction between the irrigant and canal walls.¹⁻⁴

The unique Ultrawave NiTiSonic tip is designed for continuous and intermittent passive ultrasonic irrigation, making the most important procedure in endodontics more efficient. The ultrasonic vibration of the tip allows the irrigation liquid to disrupt surface-adherent biofilms and to reach anatomic complexities within root canal systems.⁵ The NiTiSonic tip is smooth and flexible, designed to navigate the curvatures of the root without removing dentin.

SCALING
ENDO
PERIO
RESTORATIVE



The Ease of Efficiency

Don't waste time and materials by filling and refilling the tank on your piezoelectric device during your root canal treatments. The Ultrawave XS Bottle Connector allows you to connect Ultradent bottles directly to your Ultrawave XS device and use the irrigant immediately. Plus, you can easily complete the final flush with water, making sure your device is internally clean.



Step-by-Step Instructions



1. Remove water tank.



2. Place the bottle cap on the bottle of desired irrigant.



3. Connect bottle cap to the connector base.



4. Press connector base onto your Ultrawave XS device.



5. Remove connector base by pulling on the bottle.



6. Twist off the connector base and bottle cap.

4307 Ultrawave XS NiTiSonic Tip and Bottle Connector Kit

- 1 x Bottle cap with valve
- 1 x Connector base
- 1 x Empty bottle
- 1 x Ultrawave Metal Flat Wrench
- 4 x Ultrawave NiTiSonic tips



4308 Ultrawave XS Bottle Cap



F43760 Ultrawave NiTiSonic Tips (4pk)



F00446 Ultrawave Metal Flat Wrench



F58100 Ultrawave XS LED Kit

- 1 x Ultrawave XS piezoelectric device
- 1 x Ultrawave XS LED handpiece
- 3 x Ultrawave tips
- 3 x Wrenches



Also available Endo Irrigants

4612 ChlorCid® Sodium Hypochlorite Solution

1 x 480 ml bottle



4616 EDTA 18% Solution

1 x 480 ml bottle



NiTiSonic tips are designed for use with Ultradent's Ultrawave XS device only.

1-5 Data on file.