EN UltraTemp™ REZ II Temporary Resin Cement

Product Description
UltraTemp™ REZ II temporary resin cement is a low viscosity, temporary filling/milling material featuring a hydrophilic resin-based formula which is recommended for the retention of provisional restorations or provisional prostheses. UltraTemp REZ II temporary resin cement may also be used for temporary restorative procedures (i.e., provisional crowns, bridges, inlays, and onlays). It is available in Fast Set (1 to 2 minutes) and Regular Set (2 to 3 minutes).

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Forms

- 5 ml Dual Barrel Syringe
 - UltraLent Mixing Tip
 - NOTE: If expressing intraorally, preload a delivery syringe (i.e., 0.5 ml Skin-Spray™) and use intraoral delivery tip (e.g., Black Mini™ tip).
- Indications for Use/Intended Purpose**
For temporization of provisional prosthesis or restorative procedures (i.e., provisional crowns, bridges, inlays, and onlays).

Device	Set Time
UltraTemp REZ II temporary resin cement - Regular Set	2-3 minutes
UltraTemp REZ II temporary resin cement - Fast Set	1-2 minutes

Cleanup

- Remove provisional or temporary restoration.
- Remove residual cement with hand instrument.
- Clean preparation with pump-up-type slurry and cup or put UltraTemp REZ II into a container with chlorhexidine antibacterial slurry with STABrush™ intercoronal brush.
- Follow manufacturer's instructions for cementation.

Warnings and Precautions

- Resins can be sensitizing. Avoid repeated contact of uncured dental resin with eyes, skin, and soft tissue.
- Do not use on patients with known or suspected sensitivity to acrylates or other resins. If allergic reaction, dermatitis, or rash develops, have patient consult their physician.
- Do not use UltraTemp REZ II temporary resin cement with peroxide-based chemicals (i.e., walking bleach).
- For patients or users with allergy concerns, refer to product allergen information available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have patient consult their physician.

First aid instructions

- Resins can be sensitizing. Avoid repeated contact of uncured dental resin with eyes, skin, and soft tissue.
- Do not use on patients with known or suspected sensitivity to acrylates or other resins. If allergic reaction, dermatitis, or rash develops, have patient consult their physician.

Step-by-step instructions
NOTE: Periodics will negatively affect the set of UltraTemp REZ II temporary resin cement.

Preparation

- Prior to attaching mixing tip, verify uniform flow from both syringe orifices by expressing a small amount onto a mixing pad. Use caution not to cross-contaminate.

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Descrizione del prodotto
UltraTemp™ REZ II temporary resin cement is a low viscosity, temporary filling/milling material featuring a hydrophilic resin-based formula which is recommended for the retention of provisional restorations or provisional prostheses. UltraTemp REZ II temporary resin cement may also be used for temporary restorative procedures (i.e., provisional crowns, bridges, inlays, and onlays). It is available in Fast Set (1 to 2 minutes) and Regular Set (2 to 3 minutes).

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ES UltraTemp™ REZ II Cemento de resina temporal

Descripción del producto
UltraTemp™ REZ II temporary resin cement is a low viscosity, temporary filling/milling material featuring a hydrophilic resin-based formula which is recommended for the retention of provisional restorations or provisional prostheses. UltraTemp REZ II temporary resin cement may also be used for temporary restorative procedures (i.e., provisional crowns, bridges, inlays, and onlays). It is available in Fast Set (1 to 2 minutes) and Regular Set (2 to 3 minutes).

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