EN UltraTemp™ REZ II Temporary Resin Cement

Product Description
UltraTemp™ REZ II temporary resin cement is a low viscosity, thymol-free/filling material featuring a hydrophilic resin-based formula which is recommended for the retention of provisional restorations or procedural prostheses. UltraTemp REZ II temporary resin cement may also be used for temporary restorative procedures (i.e., provisional crowns, bridges, inlays, and onlays). It is available in Fast Set (1 to 2 minutes) and Regular Set (2 to 3 minutes).

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Forms

- 5 ml Dual Barrel Syringe
 - UltraLid Mixing Tip
 - NOTE: If expressing intraorally, pre-load a delivery syringe (3 cc, 0.5 ml Sani-Spray®) and use intraoral delivery tip (e.g., Black Mini™ Tip)
- Indications for Use/Intended Purpose**
For temporization of provisional prosthesis or restorative or prosthetic (i.e., provisional crowns, bridges, inlays, and onlays).

Contraindications
Do not use on patients with known sensitivities to polycrylic acid (polyacryloylate).
Do not use on patients with known sensitivity to acrylates or other resins. If allergic reaction, dermatitis, or rash develops, have patient consult their physician.
Do not use UltraTemp REZ II temporary resin cement with peroxide-based chemicals (e.g., walking bleach).

For patients or users with allergy concerns, refer to product allergic reaction information available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions
Resins can be sensitizing. Avoid repeated contact of uncured dental resin with eyes, skin, and soft tissue.
Do not cross-contaminate any restorative material which could result from repeatedly attaching the same mixing tip.
Keep away from eyes.

Step-by-Step Instructions
NOTE: Peroxide will negatively affect the set of UltraTemp REZ II temporary resin cement.

Preparation
1) Prior to attaching mixing tip, verify uniform flow from both syringe orifices by expressing a small amount onto a mixing pad. Use caution not to cross-

contaminate orifices when re-capping syringe.

- Attach mixing tip by lining up internal cap seams with syringe orifices. Twist clockwise to lock.
 - Clean, rinse, and dry preparation.
- Use – Temporary Cementation**

- Express desired amount of UltraTemp REZ II temporary resin cement into provisional restorations.
 - Seat provisional.
 - As material sets, remove all excess cement around provisional with a hand instrument.
- Use – Temporary Restoration**

- Preload delivery syringe and firmly attach delivery tip.
- Verify flow prior to applying intraorally. If resistance occurs, replace tip of syringe orifice.
- Express desired amount of UltraTemp REZ II temporary resin cement into preparation.
- After complete set, adjust occlusion if necessary.

Cleanup

- Remove provisional or temporary restoration.
- Remove residual cement with hand instrument.
- Clean preparation with pump-cure-type acrylic and cure per UltraTemp REZ II instructions.
- Apply chlorhexidine antibacterial slurry with STABrush™ intercoronal brush.
- Follow manufacturer’s instructions for cementation.

Processing

For intraoral use (delivery syringe): Delivery syringe and UltraTemp REZ II resin cement are single-use items. **Do not dilute/bare!**
Not for intraoral use, wine syringe with an intermediate level disinfectant between uses. Do not remove mixing tip between uses.

Storage and Disposal
Dispose of waste according to local rules, guidelines, and regulations.

Miscellaneous Information
Report any serious incident to the manufacturer and the competent authority.
www.ultradent.com | 1-800-552-5512 | 801-572-4200

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