



GEMINITM

810 + 980 DIODE LASER

PBM ADAPTER KIT

| Instructions for use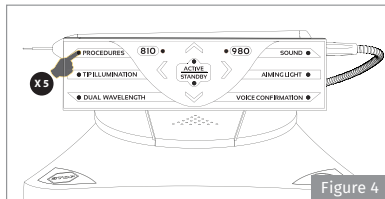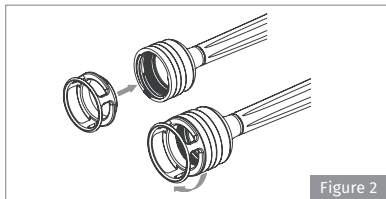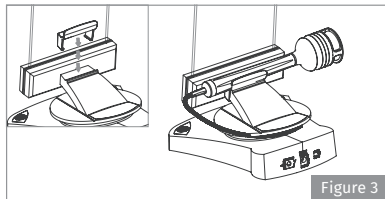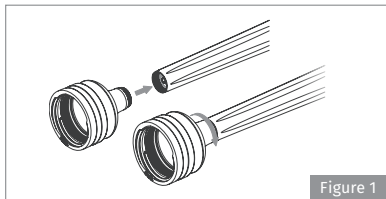

PHOTOBIMODULATION (PBM) ADAPTER

1. What's in the Box
2. Warnings & Cautions
3. Assembly
4. Software Activation & Use
5. Use Recommendations
6. Adverse Events & Contraindications
7. Maintenance
8. Specifications
9. Dose Table

SYMBOLS	DESCRIPTION
	<u>MANUFACTURER</u> INDICATES WHICH COMPANY MANUFACTURES
	<u>CATALOG PART NUMBER</u> INDICATES THE MANUFACTURER PART NUMBER
	<u>PRESCRIPTION STATEMENT</u> FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A DENTIST OR PHYSICIAN OR OTHER LICENSED MEDICAL PRACTITIONER
	<u>DISPOSABLE</u> DO NOT REUSE
	<u>INFORMATION</u> REFER TO USER'S MANUAL

1. WHAT'S IN THE BOX

- Photobiomodulation (PBM) Adapter
- Spacer (2)
- Handpiece Holder Clip
- Cleaning Cloth

- User Manual

2. WARNINGS & CAUTIONS

Failure to comply with cautions and warnings described here and in the Gemini Laser's User Manual may lead to exposure to harmful optical radiation.

Warning: The PBM Adapter must only be used with a Gemini Laser. Do not attempt to use the PBM Adapter with any other laser system or light source.

Warning: Never look directly into the PBM Adapter while the laser is active, even with safety eyewear on.

Warning: Do not use the PBM Adapter without a Spacer attached.

Caution: Do not connect or disconnect the PBM Adapter while the Gemini Laser is turned on. Only connect or disconnect the PBM Adapter when the

Gemini Laser is turned off.

Caution: Do not use any harsh chemicals or abrasives to clean the glass optics within the PBM Adapter. Doing so may damage the glass.

Caution: Do not autoclave the PBM Adapter or Spacers. Doing so will damage the components.

Caution: The Spacers are single-use only to avoid possible cross-contamination. They must be disposed of after use in a biohazard medical waste Sharps container.

Caution: Wavelength-appropriate eye protection must be worn at all times while using, and in proximity of, the PBM Adapter while it is in use. Refer to the Gemini Laser User Manual for specifics regarding proper eye protection.

3. ASSEMBLY

1. Remove the red dust covers (2) from the PBM Adapter.
2. Screw the PBM Adapter onto the end of the Gemini Handpiece until it is tight. Be careful not to cross thread the Adapter with the Handpiece. (Fig. 1)
3. Screw a Spacer onto the end of the PBM Adapter. (Fig. 2)
4. Attach the Handpiece Holder Clip to the Gemini Laser (Fig. 3)

The PBM Adapter is now ready to use.

To remove the PBM Adapter, unscrew it from the Gemini Handpiece and re-install both red dust covers when not in use.

4. SOFTWARE ACTIVATION & USE

To enable the Pain Relief preset procedure setting on your Gemini Laser:

1. Go to **www.geminiupdate.com** and follow the on-screen instructions to install the latest software on your Gemini Laser
2. Turn on the laser system
3. Before entering the passcode, press Procedures five times (Fig.4)
4. You will hear audible confirmation of "Pain Relief Active"
5. The new preset procedure is now ready for use

Note: The Pain Relief preset procedure only needs to be enabled once. Following the above steps again will disable the Pain Relief presetting.

Note: The Pain Relief preset procedure is only

available on 810 nm Wavelength mode.

To use the Pain Relief presetting:

1. Make sure the laser is in 810 nm wavelength mode
2. Navigate to Pain Relief in the preset procedure menu
3. The numbers on the display will begin to blink, prompting the selection of treatment time
4. Use the right and left arrows to adjust the desired treatment time in 3-second increments, with a maximum treatment time of 60 seconds
5. Touch the Active button to lock in the treatment time and activate the laser system
6. The laser is now ready for treatment. Depress the foot pedal to fire the laser. Keep the foot pedal depressed throughout the duration of the treatment. The laser will automatically stop firing

and return to Standby mode when the timer expires.

7. The timer reverts to the previously selected treatment time in Standby mode. To proceed to the next treatment, adjust the treatment time as needed and then press the Active button.

5. USE RECOMMENDATIONS

The intended use of the PBM Adapter is to allow the Gemini Laser to provide near-infrared laser energy to a tissue surface for the purpose of photobiomodulation.

Affected muscles and/or joints have to be exposed to an adequate level of laser energy over a period of time to provide effective results. Some cases may require more than one laser treatment, or a series

of treatments, before significant improvement is reported. Repeat the treatment as necessary and monitor the progress of the patient's condition throughout the treatment.

Diode laser wavelengths, especially 810 nm, are well absorbed in melanin in the skin, which can lead to greater heating of the target tissues in patients with darker skin types. Power and treatment time should be taken into account for patients of varying skin pigmentation. Refer to the Fitzpatrick Skin Type Scale for proper skin classification.

Pain relief preset procedure settings are programmed into the Gemini Laser for ease of use. Always use professional clinical judgement when selecting the laser settings for pain therapy. Monitor the patient and adjust the power and/or treatment

time as necessary to ensure both efficacy and patient comfort. The preset procedure setting is not meant to be a clinical recommendation in any way.

When you are ready to begin treatment, make sure a Spacer is attached and then hold the PBM Adapter in contact with the target treatment area. The PBM Adapter is designed to be held in a constant location for the duration of the treatment. If the desired treatment area is larger than the PBM Adapter's spot size of 25 mm, move the Adapter to a new location and start a new treatment only after the initial treatment time has elapsed. Always monitor patient response; adjust power and/or time as needed for patient safety and comfort.

6. ADVERSE EVENTS &

CONTRAINDICATIONS

If patient discomfort or reddening of the skin in the treatment area occurs at any time during treatment, you can do the following:

- Defocus the laser energy by moving the Adapter a few centimeters back from the skin
- Decrease the treatment time
- Stop treatment

If blistering of the skin occurs, or the patient feels a burning sensation, immediately stop treatment and rinse the area with cool water or place a cold pack on the affected area for at least 5 minutes. Afterwards, apply a burn ointment or spray. **DO NOT USE ICE.**

Do not use over articles of clothing.

Do not treat open wounds.

Do not apply ointment, creams, lotions or heating lotion patches at, or in close proximity to, the treatment area.

Do not apply therapies prior to treatment that could change body temperature, such as ultrasound, ice/heat pack, electrical stimulation, or heating patches.

Avoid treatment sites with tattoos.

Different implant materials will respond differently to laser energy and heat; be aware of any implants and their location; avoid direct exposure to laser energy or heat at the site of the implant.

Excessive fatty tissue is known to transmit heat without much attenuation, therefore increase distance or decrease treatment time.

Muscle tissue closer to the skin surface may

experience a higher absorption of heat; carefully monitor skin temperature and reduce treatment time as necessary.

Patients with swelling and/or inflammation may be sensitive to heat; reduce treatment time as necessary to ensure comfort during treatment.

Patients with tender or sensitive skin may be hypersensitive to heat; reduce treatment time as necessary to ensure comfort during treatment.

Scar tissue has been associated with poor circulation and reduced cooling through heat transport by blood; reduce treatment time as necessary to avoid overheating.

Do not treat directly over the site of any known primary malignant carcinoma or secondary metastasis

except for palliative care with informed consent and oncologist permission.

Do not treat pregnant women as the effects of photobiomodulation therapy on the fetus are unknown.

7. MAINTENANCE

The disposable Spacers are supplied non-sterile by the manufacturer and can be wiped with isopropyl alcohol wipes by the operator prior to use. The Spacers are intended for single-use only and should never be autoclaved or reused to prevent cross-contamination.

The PBM Adapter is also provided non-sterile by the manufacturer and can be wiped as needed using isopropyl alcohol wipes. Do not submerge the PBM

Adapter in any type of cleaning solution. DO NOT AUTOCLAVE the PBM Adapter.

Use the included cleaning cloth to gently wipe the glass optics as needed. Do not use any harsh chemicals or abrasives to clean the glass optics within the PBM Adapter. Doing so may damage the glass.

8. SPECIFICATIONS

Dimensions: 63 mm L x 41 mm W x 41 mm H

Weight: 76 grams

Spot Size: 25 mm diameter

Materials: Aluminum, Glass, Plastic

Shelf Life: 10 Years

Power (with Pain Relief setting selected): 1 Watt

Power Density: 204mW/cm²

Maximum Dose (time limited): 12J/cm²

Warranty: 2 Years – See Gemini Laser Warranty Card for terms and conditions

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© 2025 Ultradent Products, Inc. All rights reserved. PM-9-019 Rev E 1007333AR05 051225
Manufactured by Azena Medical, LLC. 1777 Botelho Drive, Suite 100
Distributed by Ultradent Products Inc., 505 West (10200 South) Ultradent Drive,
MedEnvoy Global B.V.