1. Remove product from packaging. 'U' • Upper whitening tray 'L' • Lower whitening tray

2. Position upper tray on teeth.

3. Bite firmly, then suck on tray for 2 seconds.

4. Remove colored outer tray, leaving the white inner tray on teeth. Repeat the process for the lower tray.

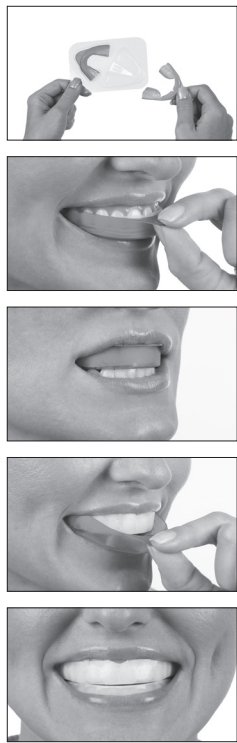
5. After indicated wear time, remove whitening trays and brush teeth.

Precautions:

- 1. Limit contact with soft tissue.
2. Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow.
3. Some patients may experience temporary discomfort of the gums, lips, throat, tongue, or teeth. Should any of these symptoms persist or worsen after discontinuing use, or in cases of severe discomfort, call your dental professional.
4. Avoid swallowing large amounts of gel. Ingestion of small amounts of gel during use is expected.
5. If the product is ingested, drink large amounts of water and contact a physician. If the product is aspirated, seek immediate medical attention.
6. Restorative materials will not whiten. We recommend whitening dark teeth before restorative placement.
7. Wait at least 7-10 days after whitening procedures before placing adhesive restorations.
8. Those with serious health concerns should consult their primary care provider prior to treatment.
9. Avoid contact with eyes. Rinse immediately if product comes into contact with them.
10. To prevent additional irritation, gingival and general oral health should be evaluated prior to initiating treatment.
11. Monitor patient's whitening progress to prevent whitening beyond the desired shade or translucency.
12. Do not eat, drink, or use tobacco products while whitening trays are in place.
13. Do not wear whitening tray while sleeping.
14. Bring treatment tray to room temperature before use.
15. Pregnant or nursing women should not use teeth whitening products.
16. For patients or users with allergy concerns, refer to product allergen document available at www.ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.
17. Contains sodium fluoride.

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