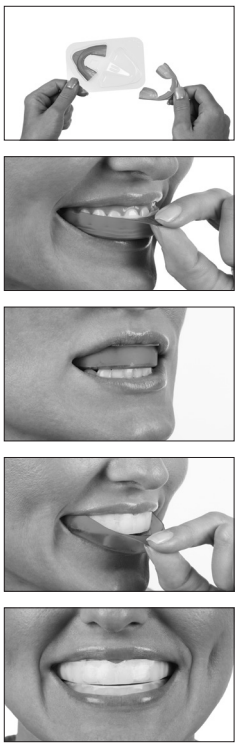
- Remove product from packaging. 'U' • Upper whitening tray 'L' • Lower whitening tray
- Position upper tray on teeth.
- Bite firmly, then suck on tray for 2 seconds.
- Remove colored outer tray, leaving the white inner tray on teeth. Repeat the process for the lower tray.
- After indicated wear time, remove whitening trays and brush teeth.

**Precautions:**

- Limit contact with soft tissue.
- Some amalgam fillings may leave a dark purple color in your whitening tray; this is normal.
- Avoid treatment if patient may swallow or aspirate tray.
- If you have any questions regarding this product, please consult your dental professional.
- Some patients may experience temporary discomfort of the gums, lips, throat, tongue, or teeth. Should any of these symptoms persist or worsen after discontinuing use, or in cases of severe discomfort, call your dental professional. Do not eat, drink, or use tobacco products while whitening trays are in place.
- Do not wear whitening tray while sleeping.
- Avoid swallowing large amounts of gel. Ingestion of small amounts of gel during use is expected.
- For patients or users with allergy concerns, refer to product allergen document available at [www.ultradent.com](http://www.ultradent.com). If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.
- Restorative materials will not whiten.
- Inform your dentist that you have been whitening prior to any dental procedure.
- Patients with serious health concerns should consult their primary care provider prior to treatment.
- Pregnant or nursing women should not use teeth whitening products.
- White spots on the enamel may become more apparent during the whitening process and often blend with continued whitening.
- Teeth can re-stain over time, do touch up teeth whitening treatments as necessary.
- To prevent additional irritation, gingival and general oral health should be evaluated prior to initiating treatment.
- Monitor progress to prevent whitening beyond the desired shade or translucency.

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