## Opal™ Bond MV Orthodontic Adhesive **EN**

**Product Description**  
Opal™ Bond MV adhesive is a medium viscosity, light curable orthodontic adhesive designed for bonding brackets and other bondable appliances to etched enamel. For all products described, carefully read and understand all instructions and SDS information prior to use.

**Delivery Form(s)**

- Dispensing gun and 0.3 g capsule
- 0.9 g syringe

**Indications for Use/Intended Purpose**

Opal Bond MV adhesive is a light cure orthodontic adhesive designed for bonding brackets and other bondable appliances to etched enamel.

- Contraindications**
- Do not use on patients with known sensitivity to methacrylates or other resins. If dermatitis, rash, or other allergic reaction is observed, rinse exposed area thoroughly and have patient consult their physician.
  - For patients or users with allergy concerns, refer to product allergen document available at [www.ultradent.com](http://www.ultradent.com). If allergic reaction is observed, rinse exposed area thoroughly with water and have patient consult their physician.

**Warnings and Precautions**

- Resins can be sensitizing. Avoid repeated contact of uncured dental resin with skin and/or soft tissue. If allergic reaction, dermatitis, or rash develops have patient consult their physician.
- Cure bonding resins completely-uncured resin materials will be deficient in physical properties, stress, and wear resistance.
- Redirect overhead light to prevent premature polymerization.
- Light activated resins are sensitive to ambient light; when not in use, ensure cap is secured onto syringe or capsule.
- Syringes/Capsules are not for intraluse use. To avoid cross contamination, use a clean gown and clean instrument.
- Patient and dental professionals must wear protective eyewear with side shields.
- If the product is ingested, drink large amounts of water and contact a physician. If the product is aspirated, seek immediate medical attention.

**Stepwise Instructions**

**Preparation – Syringe/Capsule**

- Syringe: Remove syringe cap and verify flow and consistency.
- Capsule: Load capsule into dispensing gun. Remove cap and verify flow and consistency.

**Preparation – Tooth**

- Clean enamel surfaces to be bonded with pumice or a similar water-based, mild abrasive.
- Etch, rinse, and dry all tooth surfaces prior to bonding.
- Apply a thin layer of orthodontic primer (Opal™ Seal primer) or adhesive (Peak™ Universal Bond adhesive) to the etched surface.

**Use**

- Apply a thin layer of Opal Bond MV adhesive onto the bracket pad or other bondable appliance ensuring complete coverage and mesh penetration.
- Place bracket or appliance coated with Opal Bond MV adhesive onto etched/primed tooth.
- Remove any excess uncured Opal Bond MV adhesive from circumference of bracket base or appliance.
- Light cure for 10 seconds on standard mode for curing lights with an intensity of ≥ 800 mW/cm<sup>2</sup> in the wavelength region of 385–515 nm.

- Clean up**
- Opal Bond MV adhesive capsule: single use – discard after use.
  - Opal Bond MV adhesive syringe: wipe syringe with an intermediate level disinfectant between uses.
  - Process dispensing gun according to manufacturer’s instructions.

**Processing**

To avoid cross-contamination, wipe syringe with an intermediate level disinfectant between uses and use a disposable barrier sleeve. If a barrier sleeve is not used, syringe should be treated as single-use.

**Storage and Disposal**

Dispose of waste according to local rules, guidelines, and regulations.

**Miscellaneous Information**

Report any serious incident to the manufacturer and the competent authority.

[www.ultradent.com / 1.800.552.5512 / 801.572.4200](http://www.ultradent.com / 1.800.552.5512 / 801.572.4200)

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Copyright© Производители и компетентному органу в любом из следующих стран/регионов: