




# Zirconia Luting Guide—UltraCem®

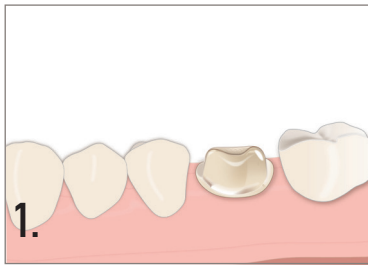


**1.**  
Clean, rinse, and dry preparation. Try on and verify fit of zirconia prosthesis.

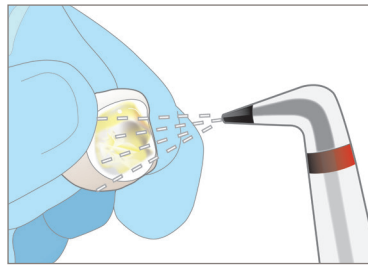


Air abrade internal surface, rinse, dry, and set aside. *NOTE: Contamination to the internal surface of the prosthesis will cause a decrease in bond strength. Keep area clean and free of phosphoric acid etch and saliva.*

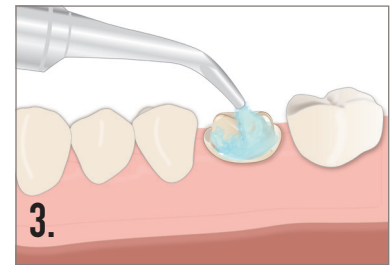


**3.**  
Clean tooth surface by applying an abrasive such as Consepsis® Scrub.

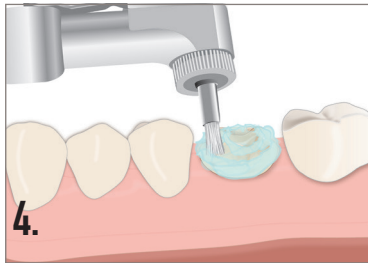


**4.**  
Scrub abrasive with the STARbrush® to clean, disinfect, and remove any residual cement. Rinse then air dry.

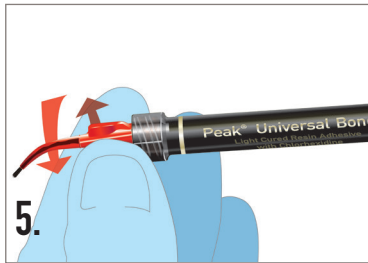


**5.**  
Securely fasten Inspirall® Brush tip to Peak® Universal Bond syringe. *Bottle delivery also available.*

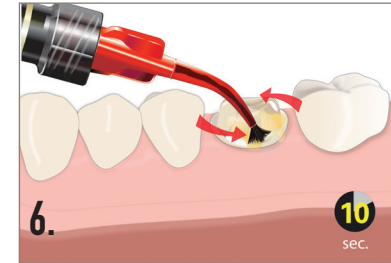

**6.**  
Apply Peak Universal Bond in a scrubbing motion for 10 seconds. (No etchant required.)

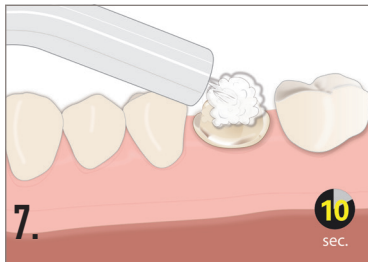


**7.**  
Aggressively air thin for 10 seconds. Surface should appear dull.

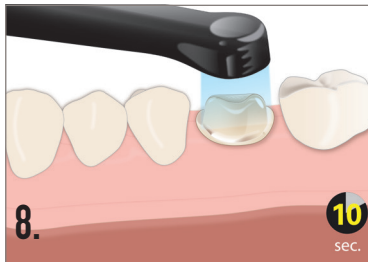


**8.**  
Light cure Peak Universal Bond for 10 seconds.



**9.**  
Place mixed UltraCem inside prosthesis.

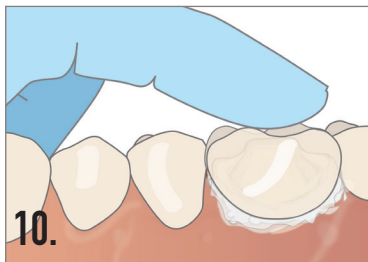

**10.**  
Seat immediately. UltraCem has a working time of 1–3 minutes at room temperature.

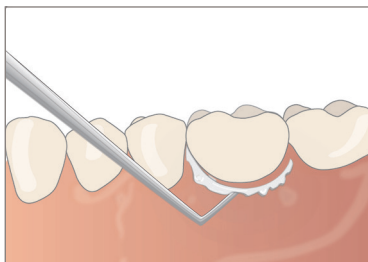

Remove excess cement when it reaches a gel-like consistency for easier cleanup. UltraCem will be fully cured within 5 minutes at 37°C.