

Dentist-designed at-**Home** whitening



Our cutting-edge technology lines the tray with whitening gel for precise placement, so patients just need to position the trays on their teeth and wear for the recommended time.

Opalescence[™]
Go
Advanced


Your patients want a simple way to get a brighter smile they are confident in. Now, you can offer them professional, mess-free tooth whitening with our new Opalescence Go Advanced prefilled trays.



Brighter Smiles. Better Days.

Opalescence™

- Professional results with one-step placement
- Prefilled Ultrafit™ tray is ready to use right out of the box, conforming to any patient's smile
- Whitening gel is distributed evenly on the tray, delivering it exactly where needed
- Whitens teeth up to 4–5 shades for quick, visible results¹

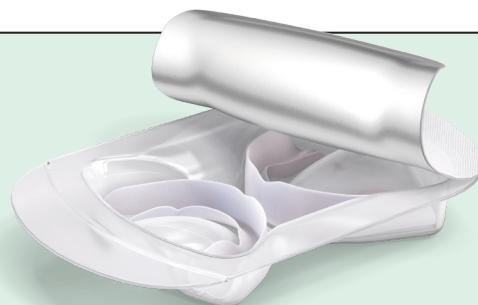

Procedure*



Scan to see the how-to video



Remove product from packaging.
'U' – Upper whitening tray
'L' – Lower whitening tray



Position upper tray on teeth.



Bite firmly, then suck on tray for 2 seconds. Do not press tray against teeth as it pushes the whitening gel against the gums, causing tissue irritation.



Repeat the process for the lower tray.



After indicated wear time, remove whitening trays and brush teeth.



10%: 30–60 min.



15%: 15–20 min.

*See IFU for complete instructions and precautions



Opalescence Go Advanced Patient Kits Case of 6

Flavor	10%	15%
Cool Mint	5821A-US	5822A-US

10 x Each upper/lower trays in each kit



Opalescence Go Advanced Mini Kits Case of 12

Flavor	10%	15%
Cool Mint	6005A-US	6006A-US

4 x Each upper/lower trays in each kit



Opalescence Go Advanced Sample Tower

Flavor	10%	15%
Cool Mint	6003A-US	6004A-US

20 x Each upper/lower trays with instructions in each sample

1. Cordeiro D, Toda C, Hanan S, et al. Clinical evaluation of different delivery methods of at-home bleaching gels composed of 10% hydrogen peroxide. Oper Dent. 2019;44(1):13–23. doi:10.2341/17-174-C.