

VALO™

BROADBAND **LED** CURING LIGHT



ECLIPSING THE COMPETITION.

VALO™ SUGGESTED CURING TIMES

DENTAL

STANDARD

HIGH POWER PLUS



XTRA POWER

POWER BUTTONS
ON FRONT AND BACK

TIMING INTERVAL LIGHTS

MODE/STATUS LIGHT

TIME/MODE
CHANGE BUTTON

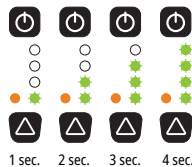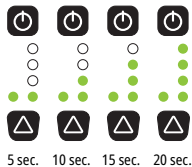

PER 2 mm LAYER

1 X 10 SECONDS

2 X 4 SECONDS

1 X 3 SECONDS


FINAL CURE

2 X 10 SECONDS

3 X 4 SECONDS

2 X 3 SECONDS

EFFECTIVE COMPOSITE-CURING
WAVELENGTH BANDS

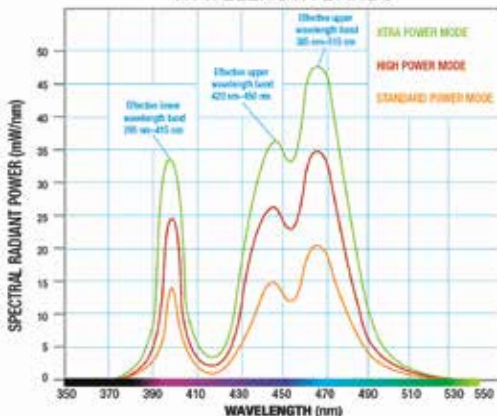

Please refer to the light cured materials' Manufacturer Instructions for Use before curing with the VALO light.

800.552.5512 | ULTRADENT.COM

© 2020 Ultradent Products, Inc. All Rights Reserved. 1004575AR03 111919

