

Intraoral Final Polishing

The blue (ultrafine) Jiffy HiShine polishers are an additional step to provide an extra smooth and highly-polished surface.

- 3061 Jiffy Ultrafine HiShine Polishing Cup—Blue
- 3062 Jiffy Ultrafine HiShine Polishing Disk—Blue
- 3060 Jiffy Ultrafine HiShine Polishing Point—Blue

Recommended speed: 5,000–7,500 RPM



Use Jiffy Ultrafine Blue HiShine Polishing cup as an additional step to create extra smooth and highly polished finish on cusp, labial, buccal, and cervical areas.



Use Jiffy Ultrafine Blue HiShine Polishing disk as an additional step to create an extra smooth and highly polished finish on labial surfaces.



Use Jiffy HiShine Blue Polishing point as an additional step to create an extra smooth and highly polished finish on occlusal and labial surfaces.

Jiffy Regular and Pointed brushes are impregnated with silicon carbide polishing particles to give an excellent final polish.

- 850 Jiffy Regular Brush
- 1009 Jiffy Pointed Brush



Use Jiffy Brushes impregnated with silicon carbide particles to create a final finish on all surfaces. For best results, apply pressure during polishing.

Recommended speed: 1,000–3,000 RPM

The unique Jiffy™ Goat Hair Brush used with Ultradent™ Diamond Polish Mint provides another option for a final esthetic finish.

- 1029 Jiffy Goat Hair Brush
- 5540 Diamond Polish Mint 0.5 μm
- 5541 Diamond Polish Mint 1 μm



Jiffy Goat Hair Brush and Ultradent Diamond Polish can be used for a final esthetic finish.

Recommended speed: 10,000 RPM



Ultradent Diamond Polish Mint.



DON'T SHORTCUT YOUR FINISH

Ultradent's Jiffy™ Composite Finishing Line provides a complete, quality solution to easily achieve a beautiful finish—without shortcut results.



Jiffy natural COMPOSITE SYSTEM

The Jiffy Natural Composite finishing and polishing systems consist of malleable spiral-shaped wheels that naturally conform to all tooth surfaces.

- Easily re-creates the luster of natural enamel
- Complete procedural kit includes everything you need
- Ultradent diamond grit gives a beautiful finish on any composite material
- Ideal for finishing Ultradent's Mosaic™ universal composite

Jiffy ORIGINAL COMPOSITE SYSTEM

The original Jiffy Composite finishing and polishing system is excellent for contouring, finishing, and polishing any composite material.

- Polishing cups feature a flared, flexible thin-wall design ideal for polishing near the gingiva
- Regular and Pointed brushes are impregnated with silicone carbide polishing particles
- Ultradent diamond grit gives a beautiful finish on any composite material

HANDPIECE SELECTION:

- Electric handpieces provide accurate speed control and consistent torque, which are important when adjusting and polishing composites
- Speed control and light hand pressure reduce heat generation and provide optimal results

Intraoral Shaping and Polishing

Use green (coarse) Jiffy cups, disks, and points for gross adjustments and shaping.

- 890 Jiffy Coarse Polishing Cup—Green
- 891 Jiffy Coarse Polishing Disk—Green
- 892 Jiffy Coarse Polishing Point—Green

Recommended speed: 7,500–10,000 RPM



Use Jiffy Coarse Green Polishing cup to shape cusp, labial, buccal, and cervical areas.



Use Jiffy Coarse Green Polishing disk to shape labial surfaces.



Use Jiffy Coarse Green Polishing point to shape occlusal and labial surfaces.

Intraoral Shaping and Polishing Continued

Use yellow (medium) Jiffy cups, disks, and points for fine shaping.

- 838 Jiffy Medium Polishing Cup—Yellow
- 840 Jiffy Medium Polishing Disk—Yellow
- 839 Jiffy Medium Polishing Point—Yellow

Recommended speed: 7,500–10,000 RPM



Use Jiffy Medium Yellow Polishing cup to polish margins and labial surfaces.



Use Jiffy Medium Yellow Polishing disk to polish occlusal and labial surfaces.



Use Jiffy Medium Yellow Polishing point to polish margins and labial surfaces.

Jiffy Natural yellow (medium) wheels are malleable spiral-shaped wheels designed to naturally and easily conform to all tooth surfaces, including occlusal.

- 6089 Jiffy Natural RA Medium 14 mm Polishing Wheel—Yellow

Recommended speed: 5,000–8,000 RPM



Use Jiffy Medium Yellow Natural wheels to polish on all areas except near the gingiva, where a Jiffy Medium Polishing cup should be used.



Use white (fine) Jiffy cups, disks, and points for fine polishing.

- 841 Jiffy Fine Polishing Cup—White
- 843 Jiffy Fine Polishing Disk—White
- 842 Jiffy Fine Polishing Point—White

Recommended speed: 5,000–7,000 RPM



Use Jiffy Fine White Polishing cup to create final polish on cusp, labial, buccal, and cervical areas.



Use Jiffy Fine White Polishing disk to create final polish on labial surfaces.



Use Jiffy Fine White Polishing point to create final polish on occlusal and labial surfaces.

Jiffy Natural white (fine) wheels are malleable, spiral-shaped wheels designed to naturally and easily conform to all tooth surfaces, including occlusal.

- 6090 Jiffy Natural RA Fine 14 mm Polishing Wheel—White

Recommended speed: 2,000–4,000 RPM



Use Jiffy Fine White Natural wheels to create final polish on all areas except near the gingiva, where a Jiffy Fine White Polishing cup should be used.

