UltraTemp™ Temporary Cement	EN
--------------------------------	----

Product Description
UltraTemp™ is a non-eugenol temporary cement. It is polycarboxylate based and does not negatively affect resin bonding.
For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Forms

- 5 ml Dual Barrel Syringe
- UltraTemp Mixing Tip
- NOTE: If expressing intraorally, preload a delivery syringe (e.g., 0.5 ml Skini syringe) and use intraoral delivery tip (e.g., Black Mini™ Tip)

Indications for Use/Intended Purpose

UltraTemp is a non-eugenol temporary cement indicated for interim cementation of inlays, onlays, crowns, and bridges. It is polycarboxylate based and does not negatively affect resin bonding. Being water soluble until set, it cleans up easily.

Contraindications

Do not use on patients with known sensitivities to polyacrylic acid (polycarboxylate).
For patients or users with allergy concerns, refer to product allergen document available at ultraid.com. If allergic reaction is observed, rinse exposed area thoroughly with water and have the patient consult their physician.

Warnings and Precautions

- Avoid cross-mixing of syringe orifices by leaving mixing tip attached between uses
- To avoid cross-contamination use clean gloves and a clean instrument
- Syringes are not for intraoral use
- Keep away from eyes
- Patient and dental professionals must wear protective eyewear with side shields
- Use only recommended tips
- Tips are single use only

Stepwise Instructions

- For attaching mixing tip, verify uniform flow from both syringe orifices by expressing a small amount onto a mixing pad. Use caution not to cross-mix orifices when re-capping syringe.
 - Attach mixing tip by lining up internal cap stems with syringe orifices. Twist clockwise to lock.
 - For dual barrel: Not for intraoral use. Preload syringe with UltraTemp, attach delivery tip, and verify flow, color, and consistency prior to express intraorally.
 - Clean, rinse, and dry preparation.
- Use – Temporary Cementation**
- Express a small amount of UltraTemp temporary cement into provisional.
 - Seat provisional.
 - As material sets, remove all excess cement around provisional with a hand instrument.

Device	Set Time
UltraTemp temporary cement	2–3 minutes

Cleanup
Provisional:

- Remove provisional or temporary restoration.
- Remove residual cement with hand instrument.
- Clean preparation with pumice-type slurry and cup or Ultradent Consepss™ Scrub chlorhexidine antibacterial slurry with STARbrush™ interconoral brush.
- Follow manufacturer’s instructions for cementation.

Processing

For intraoral use (delivery syringe): Delivery syringe and delivery tip are considered single use.

For dual barrel: Not for intraoral use. Wipe syringe with an intermediate level disinfectant between uses. Do not remove mixing tip between uses.

Storage and Disposal
Store according to outer package label.
Dispose of waste according to local rules, guidelines, and regulations.
Miscellaneous Information
Report any serious incident to the manufacturer and the competent authority.
ultrad.com / 1.800.552.5512 / 801.572.4200

UltraTemp™ Midtlerdig cement	DA
---------------------------------	----

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