

IRRIGATION PROTOCOL

RECOMMENDED IRRIGATION PROTOCOL



1. Establish Patency:

- Fill canal with ChlorCid™ V (NaOCl) solution and place a small amount of File-Eze™ EDTA lubricant
- Scout canal with a #10 or #15 SS K-file until the temporary working length and irrigate with 3–5 ml of ChlorCid™ solution



2. Preflaring:

- Fill canal with ChlorCid V solution and use orifice shaper in the straight part of canal
- Irrigate with 3–5 ml of ChlorCid solution



3. Glide Path:

- Fill canal with ChlorCid V solution and create a glide path with a #15 or #20 SS K-file until working length
- Irrigate with 3–5 ml of ChlorCid solution



4. Instrumentation:

- Fill canal with ChlorCid V solution, then start instrumentation
- Irrigate with 3–5 ml of ChlorCid solution between each file
- Finish irrigation with 3–5 ml of ChlorCid solution
- Fill canal with EDTA, leave 60 seconds, and evacuate



5. Optional Step:

- Rinse canal with sterile water, and soak canal for 5 minutes with Consepsis™ (CHX) solution
- Evacuate canal and dry; do not rinse

DO NOT mix chlorhexidine and NaOCl due to toxic chemical formation.

800.552.5512 | ULTRADENT.COM

© 2020 Ultradent Products, Inc. All Rights Reserved.

1006274AR04 031020



IRRIGATION PROTOCOL

Рекомендуемый Протокол Ирригации

Обеспечьте Проходимость:



- Заполните канал раствором ChlorCid™ V (NaOCl) и нанесите небольшое количество геля File-Eze™ EDTA
- Исследуйте канал К-файлом (SS) #10 или #15 до ориентировочной рабочей длины и промойте 3-5 мл раствора ChlorCid™

Подготовка:



- Заполните канал раствором ChlorCid V и используйте устьевой файл в прямых частях канала
- Промойте 3-5 мл раствора ChlorCid

“Ковровая дорожка” (Glide Path):



- Заполните канал раствором ChlorCid V и создайте “Ковровую дорожку” К-файлом (SS) #15 или #20 до рабочей длины
- Промойте 3-5 мл раствора ChlorCid

Инструментальная обработка



- Заполните канал раствором ChlorCid V, а затем начните инструментальную обработку
- Промойте 3-5 мл раствора ChlorCid после каждого файла
- Закончите обработку промыванием 3-5 мл раствора ChlorCid
- Заполните канал ЭДТА, оставьте на 60 секунд и уберите

Дополнительный Этап:



- Промойте канал стерильной водой и заполните раствором ConsepSis™ (CHX) в течение 5 минут
- высушите канал; не смывайте

800.552.5512 | ULTRADENT.COM

© 2020 Ultradent Products, Inc. All Rights Reserved.

1006274AR04 031020

