

Opalescence[®]
tooth whitening systems



Every Smile Brightened!



800.552.5512 ULTRADENT.COM

Opalescence® Tooth Whitening Reference Guide



Opalescence® PF



Custom Whitening Tray
(Requires Impression)

Opalescence® PF 10%



Wear Time:
8–10 hours per day



Flavors:
Mint, Melon, and Regular

Active Ingredient:

10% carbamide peroxide

Contains:

Potassium nitrate and fluoride

Indications:

For patients with existing sensitivity; can be worn day or night

Opalescence® PF 15% and 20%



Wear Time:
15%: 4–6; 20%: 2–4 hours per day



Flavors:
Mint, Melon, and Regular

Active Ingredient:

15% and 20% carbamide peroxide

Contains:

Potassium nitrate and fluoride

Indications:

For faster whitening; can be worn during the day

Opalescence® PF 35%



Wear Time:
30–60 minutes a day



Flavors:
Mint, Melon, and Regular

Active Ingredient:

35% carbamide peroxide

Contains:

Potassium nitrate and fluoride

Indications:

For shorter wear time and touch-ups

Opalescence® PF 45%



Wear Time:
15–30 minutes



Flavors:
Mint

Active Ingredient:

45% carbamide peroxide

Contains:

Potassium nitrate and fluoride

Indications:

For shorter wear times



Opalescence® Go®



UltraFit™ Ready-To-Go Tray

Opalescence Go® 10%



Wear Time:
30–60 minutes per day



Flavors:
Mint, and Melon

Active Ingredient:

10% hydrogen peroxide

Contains:

Potassium nitrate and fluoride

Indications:

For fast whitening ready-to-go; an alternative to OTC products

Opalescence Go® 15%



Wear Time:
15–20 minutes per day



Flavors:
Mint

Active Ingredient:

15% hydrogen peroxide

Contains:

Potassium nitrate and fluoride

Indications:

For fast whitening ready-to-go; an alternative to OTC products



Opalescence® Endo



Applied In-Office

Opalescence® Endo



Wear Time:
3–5 days per application

Active Ingredient:

35% hydrogen peroxide

Indications:

Dentist administered for internal whitening of non-vital teeth



Opalustre®



Applied In-Office

Opalustre® Microabrasion Slurry



Wear Time:
1 or more doctor-administered applications as needed

Active Ingredient:

6.6% hydrochloric acid

Indications:

Dentist-administered chairside treatment; removes superficial white and brown stains



Opalescence® Boost®



Applied In-Office

Opalescence® Boost®



Wear Time:
Two to three 20-minute professionally administered treatments per visit (do not exceed three per visit)

Active Ingredient:

40% hydrogen peroxide

Contains:

Potassium nitrate and fluoride

Indications:

Dentist-administered, fast chairside treatment



UltraEZ®



UltraFit™ Ready-To-Go Tray



Custom Whitening Tray
(Requires Impression)

UltraEZ®



Wear Time:
15–60 minutes per day



Flavors:
Regular

Active Ingredient:

3% potassium nitrate, .25% neutral NaF

Indications:

Treats sensitivity