

Gentle on teeth, brilliant on smiles.



#1-selling
professional teeth
whitening brand



Dentist-designed
at-home whitening



Safely
whitens teeth



Brilliant options
for sensitive teeth

Visit opalescence.com to learn more

VEGAN • CERTIFIED GLUTEN-FREE • CERTIFIED KOSHER

 Manufactured by Ultradent Products, Inc. 505 West Ultradent Drive (10200 South), South Jordan, Utah 84095. Made in the USA from globally sourced components. For an SDS of this product, please see ultradent.com. ©2026 Ultradent Products, Inc. All rights reserved. 1013995AR01 030926



Opalescence™ Go™ Advanced gives you fast, professional teeth whitening results that are seriously effective. The prefilled UltraFit™ trays are ready to go straight out of the box and conform precisely to your teeth, ensuring the whitening gel stays exactly where it belongs. Glow beyond your expectations and join the 150 million smiles brightened worldwide.

Product Description

Opalescence Go Advanced tooth whitening is a take-home whitening system dispensed in pre-loaded disposable trays. It is a clear, flavored, high viscosity, sticky hydrogen peroxide gel with potassium nitrate and sodium fluoride.

For all products described, carefully read and understand all instructions and SDS information prior to use.

Delivery Form: UltraFit™ Prefilled Disposable Tray

Miscellaneous Information

- Hypocalcified areas (white spot lesions), which may not be visible to the naked eye, will whiten faster, thereby becoming more obvious during whitening. Continue whitening treatment until the unaffected tooth surface blends. Re-evaluate two weeks after whitening treatment when tooth color has stabilized.
- For a small percentage of the population, their teeth will not appreciably whiten.
- Some amalgam fillings may leave a dark purple color in your whitening tray. This is normal.
- Teeth can re-stain over time. If so, do touch up teeth whitening treatments as necessary.

Storage and Disposal: Store according to outer package label. Dispose of waste according to local rules, guidelines, and regulations.

Consult your healthcare professional for any product-related questions.

Report any serious incident to the manufacturer and the competent authority. ultradent.com / 800.552.5512 / 801.572.4200



See
instructions



Keep away from
heat/sunlight



Store in a
cool place



Do not re-use
to avoid cross
contamination



Keep out
of reach of
children



Achieve a radiant,
confident smile
that lights up your
world without any
gimmicks with
Opalescence Go
Advanced prefilled
whitening trays.



Opalescence™

Go

Advanced

Instructions:



Remove product from packaging.
'U' – Upper whitening tray
'L' – Lower whitening tray



Position upper tray on teeth.



Bite firmly, then suck on tray for 2 seconds. Do not press tray against teeth as it pushes the whitening gel against the gums, causing tissue irritation.



Repeat the process for the lower tray.



After indicated wear time, remove and discard whitening trays and brush teeth.

Wear Times

- 10%: 30–60 min
- 15%: 15–20 min



Scan QR Code
for personalized
tips from a dentist

Tips for Sensitivity

Opalescence™ whitening products are made to be gentle on teeth. If you experience sensitivity when whitening, here are some tips:

TIP 1:

Do not press the tray against teeth as it pushes the whitening gel against the gums, causing tissue irritation. Clean any excess gel from the trays that gets onto your gums with a soft toothbrush or tissue. Do not worry if the gel doesn't come in contact with your teeth near the gumline, the peroxide will whiten the entire tooth.

TIP 2:

Use Opalescence™ Whitening Toothpaste Sensitivity Relief for two weeks before whitening. After brushing, spit but don't rinse your teeth with water. The toothpaste contains potassium nitrate to help soothe sensitive teeth.

OVERALL:

Ongoing tooth sensitivity might indicate an underlying concern. Consult with your dentist before you start your whitening regimen.

TIP 3:

Try a lower hydrogen peroxide concentration instead. Sometimes, using a lower concentration for a longer period of time can reduce sensitivity.

TIP 4:

Take days off from teeth whitening. If you feel too much sensitivity using the trays daily, consider whitening every other day, every two days, or even on a weekly basis.

TIP 5:

Consider using over-the-counter oral pain relief products or topical vitamin E oil to alleviate gum-related discomfort.

Precautions

- Limit contact with soft tissue
- Avoid direct contact of the active surface of the tooth whitening gel with the gums and/or salivary flow
- You may experience temporary discomfort of the gums, lips, throat, tongue, or teeth. Should any of these symptoms persist or worsen after discontinuing use, or in cases of severe discomfort, call your dental professional.
- Avoid swallowing large amounts of gel. Ingestion of small amounts of gel during use is expected.
- If the product is ingested, drink large amounts of water and contact a physician. If the product is aspirated, seek immediate medical attention.
- Restorative materials will not whiten. We recommend whitening discolored teeth before restorative placement.
- Wait at least 7–10 days after whitening procedures before any dental restorative work is done
- Those with serious health concerns should consult their primary care provider prior to treatment
- Avoid contact with eyes. Rinse immediately if product comes into contact with them.
- To prevent additional irritation, gingival and general oral health should be evaluated prior to initiating treatment
- Monitor whitening progress to prevent whitening beyond the desired shade or translucency
- Do not eat, drink, or use tobacco products while whitening trays are in place
- Do not wear whitening tray while sleeping
- Bring treatment tray to room temperature before use
- Pregnant or nursing women should not use teeth whitening products
- For users with allergy concerns, refer to product allergen document available at ultradent.com. If allergic reaction is observed, rinse exposed area thoroughly with water and consult your physician.
- Contains sodium fluoride