



How Dry Is Your Working Environment?

OraSeal® & DermaDam®

Caulking and Putty Material

Dental Dam

OraSeal Caulking and Putty materials effectively adhere to a wet dental dam. Even under water or saliva, OraSeal materials excel at preventing leakage during treatment. In conjunction with the powder-free, tear resistant DermaDam® dental dam, OraSeal materials make it easy to achieve a dry working environment.




ULTRADENT
PRODUCTS, INC.

800.552.5512 ULTRADENT.COM

OraSeal®

Caulking and Putty Material

Achieve a Dry Working Environment

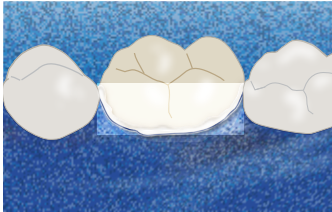

Apply OraSeal Caulking material with the Black Mini® or White Mac™ delivery tip to prevent leakage of rubber dam during endodontic treatment.* Shape with wet gloved finger, wet cotton swab, or instrument. Root canal is performed in a clean, dry field.



Use OraSeal Putty material to block out large interproximal spaces for easy and distortion-free removal of impression.



Use to ensure dental dam seal when using strong peroxide for vital whitening or porcelain etching with hydrofluoric acid. Express OraSeal Caulking material into embrasures and onto the gingiva. Stretch and position dam onto frame.

*Cohen S, Burns RC. *Pathways of the Pulp*. 7th ed. St. Louis, MO: Mosby-Year Book; 1998:123-124.

352

OraSeal Kit

2 x 1.2 ml OraSeal Caulking syringes
2 x 1.2 ml OraSeal Putty syringes
4 x Black Mini tips
20 x White Mac tips



351
353

OraSeal Caulking Refill OraSeal Putty Refill

4 x 1.2 ml syringes



354
355

OraSeal Caulking Econo Refill OraSeal Putty Econo Refill

20 x 1.2 ml syringes



DermaDam®

Rubber Dam

DermaDam rubber dam is made from pure latex rubber and is powder free, which reduces the possibility of latex reactions. Quality processing ensures a low content of surface proteins.

- Low dermatitis potential
- Strong and tear resistant
- Powder free to reduce allergic reactions

DermaDam® Synthetic

Dental Dam

DermaDam Synthetic dental dam is not made with natural rubber latex, but is designed to be just as flexible and durable as dams that are composed of natural rubber latex.



311

DermaDam Medium - 0.20 mm Refill

36 x 6" x 6" (15 cm x 15 cm) DermaDam Medium



314

DermaDam Heavy - 0.25 mm Refill

36 x 6" x 6" (15 cm x 15 cm) DermaDam Heavy



299

DermaDam Synthetic - 0.20 mm Refill

20 x 6" x 6" (15 cm x 15 cm) DermaDam Medium

