Thermo Clone™ Tray Adhesive

UP ULTRADENT
PRODUCTS, INC.

37396/0316

English	2	Norsk	12
Deutsch	3	Ελληνικά	13
Français	4	Русский	14
Nederlands	5	Polski	15
Italiano	6	Türkçe	16
Español	7	17	العربية
Português	8	한국어	18
Svenska	9	中文	19
Dansk	10	ไทย	20
Suomi	11	日本語	21

Thermo Clone™ Tray Adhesive

Adhesive for Thermo Clone™ VPS Im- pression Materials

 **ULTRADENT**
PRODUCTS, INC.

37396/0316

Description/Indications:

Thermo Clone™ Tray Adhesive is indicated as a universal tray adhesive for Thermo Clone™ VPS Impression Materials. It is suitable for use on metal and acrylic trays.

1. Use of the tray adhesive is very important. For an accurate impression, coat tray with a thin layer including the edge.
2. Let adhesive air dry for 5 minutes prior to making the impression.
3. If using plastic impression trays make sure that the adhesive is compatible with your tray.
4. Tightly recap the bottle immediately after each use to avoid evaporation of the chemical.
5. Make sure to add the impression material to the coated tray within 30 minutes of application of the adhesive.
6. The impression material can be removed from the tray with a cleaning solvent, petroleum, ether, or orange oil if needed.

Warning:

Do not use the Thermo Clone™ adhesive with condensation-curing silicones, polyether, or polysulfide impression materials.

Precautions:

Do not swallow, but if swallowed consult a physician if problems arise.
Avoid contact with the eyes; if contact happens, rinse immediately and thoroughly with water and consult a physician.
Avoid contact with the skin and clothing.
Keep away from ignition sources.
Do not use after the expiration date.
Dispose of this chemical as per local

environmental regulations.

Keep off clothing as it cannot be removed by dry cleaning.

Side Effects:

Although there are no known systemic side effects, specific contact allergies cannot be ruled out in rare cases.

Do not use the material in case of known allergy to one of the ingredients.

Composition:

Silicone resin, solvent (ethyl acetate, xylene).

Storage Conditions:

Store in a dry place at room temperature; do not expose to sunlight and avoid exposure to extreme temperature fluctuations.

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Ultradent Products, Inc
Ultradent Products GmbH