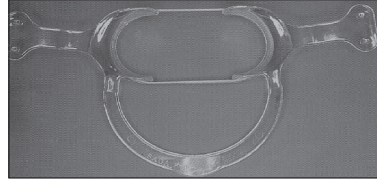
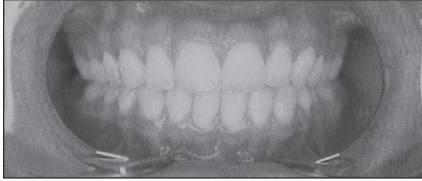
KleerView™

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EN

Instruct the patient to open his/her mouth. Insert the intraoral extension of each cheek retractor against the inside of the patient's cheeks. Gently guide the patient's lips to fit comfortably in the curve of each lip retractor. To remove KleerView from the mouth, carefully grasp the left and right cheek handles. Perfect for lingual bonding with excellent view for upper arch.

General cleaning before sterilization: use a mild cleanser and soft brush or sponge, clean and rinse KleerView of any debris, blood or saliva before cold sterilization. Place elastic band around upper/lower cheek and lip retractor to reduce tension.

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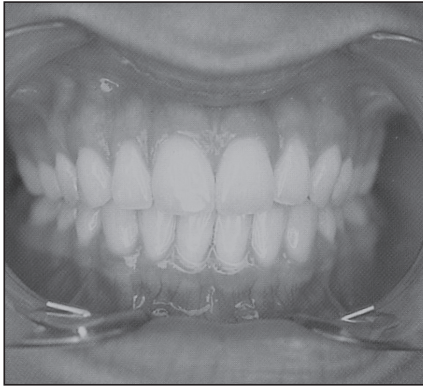
Fig. 1

U.S. federal law restricts this device to sale by or on the order of a dentist.

For immediate order and/or complete descriptions of Ultradent's product line, refer to Ultradent's catalog or call toll free: 1-800-552-5512.

Outside U.S. call (801) 572-4200, or visit www.ultradent.com.

R_x

EN- Prescription use only
 DE- Rezeptpflichtig
 FR- Utilisation uniquement par prescription
 NL- Uitsluitend verkrijgbaar op recept
 IT- Soggetto a prescrizione medica
 ES- Uso exclusivo bajo receta
 PT- Utilização apenas por prescrição
 SV- Receptbelagt
 DA- Receptpligtigt
 FI- Reseptilääke
 EL- Μόνο με συνταγή
 RU- Только для профессионального

EN- Consult instructions for use
 DE- Gebrauchsanleitung beachten
 FR- Consultez le mode d'emploi
 NL- Raadpleeg de gebruiksaanwijzing
 IT- Consultare le istruzioni per l'uso
 ES- Consulte las instrucciones de uso
 PT- Consultar as instruções de utilização
 SV- Se bruksanvisningen
 DA- Se brugsanvisningen
 FI- Lue käyttöohjeet
 EL- Συμβουλευτείτε τις οδηγίες χρήσης
 RU- См. инструкцию

EN- Manufacturer
 DE- Hersteller
 FR- Fabricant
 NL- Fabrikant
 IT- Fabbricante
 ES- Fabricante
 PT- Fabricante
 SV- Tillverkare
 DA- Producent
 FI- Valmistaja
 EL- Κατασκευαστής
 RU- Производитель

KleeView™

Cheek and Lip Retractor

Ultradent Products GmbH
 Am Westhoyer Berg 30
 51149 Cologne
 Germany

EN- Recycle
 DE- Recyceln
 FR- Recycler
 NL- Recyclen
 IT- Riciclare
 ES- Reciclar
 PT- Reciclar
 SV- Återvinna
 DA- Genbrug
 FI- Kierrätyskesä
 EL- Ανακυκλώστε το
 RU- Рециркулировать

KleeView™

Cheek and Lip Retractor

Very flexible and easy to use!

Indications:

To achieve working area accessibility, KleeView's transparent material is ideal for clinical photography.

Instructions:

1. To facilitate patient comfort, apply lip balm or petroleum jelly to the patient's lips prior to inserting KleeView cheek retractors.
2. Hold both ends and squeeze the retractors together, gently bending the connecting bow in the cheek retractor.
3. The bow is positioned at the patient's chin and provides necessary tension to hold the device in place.
4. Instruct the patient to open his/her mouth. Insert the intraoral extension of each cheek retractor against the inside of the patient's cheeks. Gently guide the patient's lips to fit comfortably in the curve of each lip retractor.
5. To remove KleeView from the mouth, carefully grasp the left and right cheek handles. Perfect for lingual bonding with excellent view for upper arch.

Cleaning/Care

General cleaning before sterilization: use a mild cleanser and soft brush or sponge, clean and rinse KleeView of any debris, blood or saliva before placing in a cold disinfection solution. Place elastic band around upper/lower cheek and lip retractor to reduce tension.

- Store at room temperature out of direct sunlight.
- Plastic fatigues over time and daily use can cause breakage.
- Do not place in refrigerator.
- Some chemicals and cleansers may make the retractor brittle and cloudy.
- Do not dry heat sterilize or steam sterilize.

For product SDS please see our website: www.ultradent.com
 1-800-552-5512; 801-572-4200

ULTRADENT
 PRODUCTS, INC.

© Copyright 2018 Ultradent Products, Inc. All Rights Reserved.
 31905.09 110518